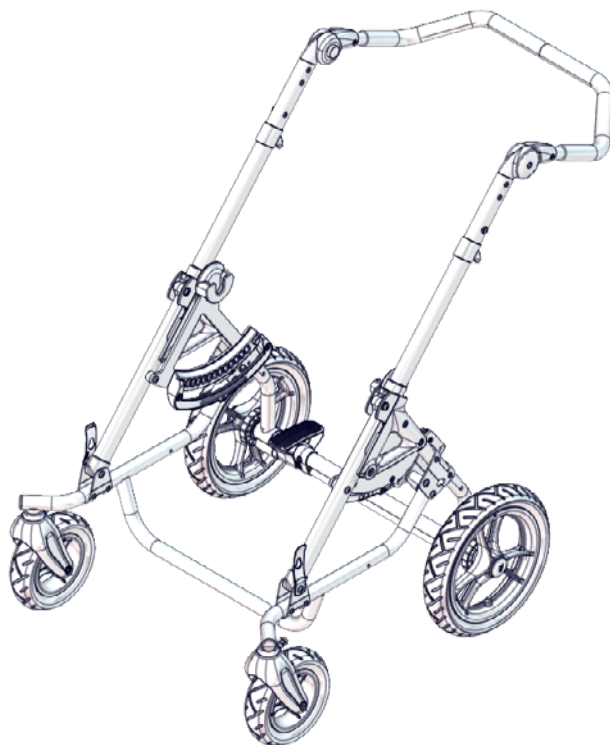
# User manual

## Chilla

**R82<sup>®</sup>**   
by Etac

M1615 - 2025-06-04- Rev. 3.0

[www.etac.com](http://www.etac.com)

Content

7

8-13

14

**mm**  
**(inch)**  
**kg**  
**(lb)**

**Frame size 1 & 2**

**15-17**

**Seat size 1 & 2**

**18-19**

20

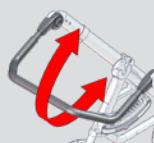
21


22


## Content


23

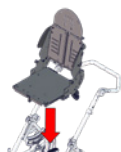
24

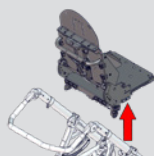25

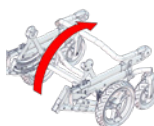26-27

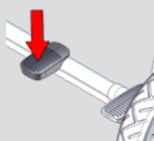28

29


30-31


---

## Content

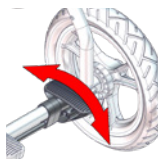---

32

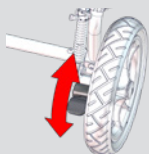33

34-35

36

37

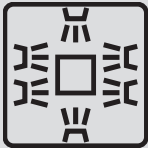
38-39

40-41

Content

44-45

46

47

48-49

## LANGUAGE

|                                 |         |
|---------------------------------|---------|
| ENGLISH (Original instructions) | 50-54   |
| DEUTSCH                         | 55-60   |
| NEDERLANDS                      | 61-66   |
| DANSK                           | 67-71   |
| NORSK                           | 72-76   |
| SVENSKA                         | 77-81   |
| SUOMI                           | 82-86   |
| ESPAÑOL                         | 87-92   |
| FRANÇAIS                        | 93-98   |
| ITALIANO                        | 99-104  |
| PORTUGUÊS                       | 105-110 |
| PORTUGUÊS DO BRASIL             | 111-116 |
| РОССИЯ                          | 117-122 |
| POLSKI                          | 123-128 |
| ČESKY                           | 129-133 |
| 中文                              | 134-138 |
| 日本語                             | 139-144 |
| ΕΛΛΗΝΙΚΑ                        | 145-150 |
| ÍSLENSKA                        | 151-155 |

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

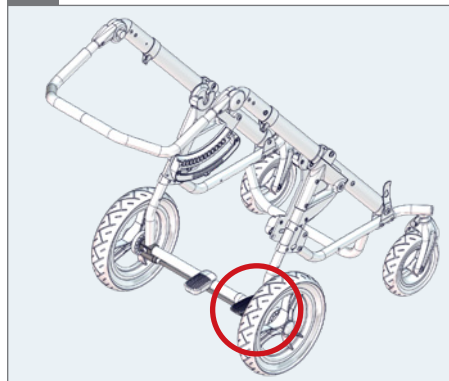
### Size 1 & 2 - basic

### Size 1 & 2 - Adjustable suspension
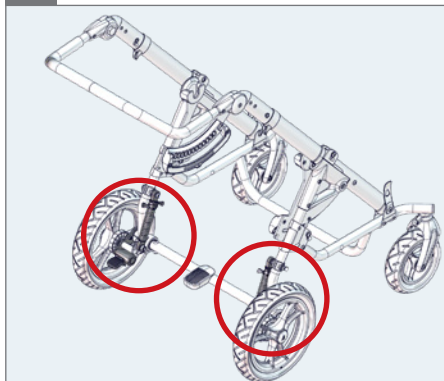
### Size 1 & 2 - Driving brakes & adjustable suspension
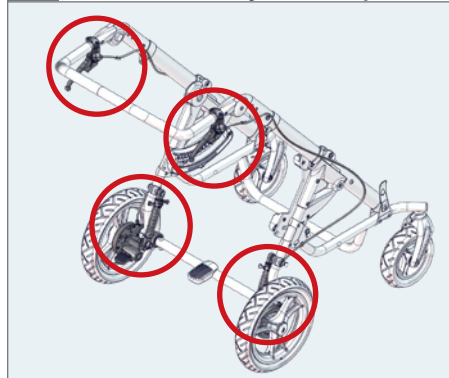
## Frame size 1 & 2

|                                                                                     |                                                   | mm (inch), kg (lb)                                                                       |                          |                                                                                          |                          |
|-------------------------------------------------------------------------------------|---------------------------------------------------|------------------------------------------------------------------------------------------|--------------------------|------------------------------------------------------------------------------------------|--------------------------|
|                                                                                     |                                                   | Size 1  |                          | Size 2  |                          |
|                                                                                     |                                                   | Normal                                                                                   | Suspension               | Normal                                                                                   | Suspension               |
|    | Overall height, to pushbrace                      | 1200 mm<br>(47¼ inch)                                                                    | 1200 mm<br>(47¼ inch)    | 1200 mm<br>(47¼ inch)                                                                    | 1200 mm<br>(47¼ inch)    |
|    | Seat surface height at front edge                 | 450 mm<br>(17¾ inch)                                                                     | 450 mm<br>(17¾ inch)     | 450 mm<br>(17¾ inch)                                                                     | 450 mm<br>(17¾ inch)     |
|    | Seat surface height at front edge (adapter plate) | 510 mm<br>(20 inch)                                                                      | 510 mm<br>(20 inch)      | 510 mm<br>(20 inch)                                                                      | 510 mm<br>(20 inch)      |
|    | Length                                            | 730 mm<br>(28¾ inch)                                                                     | 760 mm<br>(30 inch)      | 730 mm<br>(28¾ inch)                                                                     | 760 mm<br>(30 inch)      |
|    | Overall length                                    | 800 mm<br>(31½ inch)                                                                     | 800 mm<br>(31½ inch)     | 800 mm<br>(31½ inch)                                                                     | 800 mm<br>(31½ inch)     |
|   | Overall length with leg rest                      | 1180 mm<br>(46½ inch)                                                                    | 1180 mm<br>(46½ inch)    | 1180 mm<br>(46½ inch)                                                                    | 1180 mm<br>(46½ inch)    |
|  | Overall width                                     | 600 mm<br>(23½ inch)                                                                     | 620 mm<br>(24½ inch)     | 660 mm<br>(26 inch)                                                                      | 680 mm<br>(26¾ inch)     |
|  | Leg to seat surface angle                         | (-45°)-(-40°)                                                                            | (-45°)-(-40°)            | (-45°)-(-40°)                                                                            | (-45°)-(-40°)            |
|  | Front wheel h×w                                   | 190x50 mm<br>(7½x2 inch)                                                                 | 190x50 mm<br>(7½x2 inch) | 190x50 mm<br>(7½x2 inch)                                                                 | 190x50 mm<br>(7½x2 inch) |

\*380 mm (15 inch) without seat and rear wheels

\*\*with solid wheels

\*\*\*frame without rear wheels

## Frame size 1 & 2

|                                                                                     |                                     | mm (inch), kg (lb)                                                                       |                            |                                                                                          |                            |
|-------------------------------------------------------------------------------------|-------------------------------------|------------------------------------------------------------------------------------------|----------------------------|------------------------------------------------------------------------------------------|----------------------------|
|                                                                                     |                                     | Size 1  |                            | Size 2  |                            |
|                                                                                     |                                     | Normal                                                                                   | Suspension                 | Normal                                                                                   | Suspension                 |
|    | Rear wheel h x w                    | 290x60 mm<br>(11½x2¼ inch)                                                               | 290x60 mm<br>(11½x2¼ inch) | 290x60 mm<br>(11½x2¼ inch)                                                               | 290x60 mm<br>(11½x2¼ inch) |
|    | Folded height*                      | 380 mm<br>(15 inch)                                                                      | 380 mm<br>(15 inch)        | 380 mm<br>(15 inch)                                                                      | 380 mm<br>(15 inch)        |
|    | Folded length                       | 1035 mm<br>(40¾")                                                                        | 1035 mm<br>(40¾")          | 1035 mm<br>(40¾")                                                                        | 1035 mm<br>(40¾")          |
|    | Minimum turning radius              | 1250 mm<br>(49¼ inch)                                                                    | 1250 mm<br>(49¼ inch)      | 1250 mm<br>(49¼ inch)                                                                    | 1250 mm<br>(49¼ inch)      |
|    | Maximum load                        | 43 kg<br>(94 lb)                                                                         | 43 kg<br>(94 lb)           | 60 kg<br>(132 lb)                                                                        | 60 kg<br>(132 lb)          |
|    | Mass device**                       | 9.2 kg<br>(20 lb)                                                                        | 9.7 kg<br>(21 lb)          | 9.5 kg<br>(20 lb)                                                                        | 10 kg<br>(22 lb)           |
|  | Maximum rated load                  | 54.2 kg<br>(119 lb)                                                                      | 54.7 kg<br>(120 lb)        | 71.5 kg<br>(157 lb)                                                                      | 72 kg<br>(158 lb)          |
|  | Minimum user mass in transportation | 22 kg<br>(48 lb)                                                                         | 22 kg<br>(48 lb)           | 22 kg<br>(48 lb)                                                                         | 22 kg<br>(48 lb)           |
|  | Maximum user mass in transportation | 35 kg<br>(77 lb)                                                                         | 35 kg<br>(77 lb)           | 50 kg<br>(110 lb)                                                                        | 50 kg<br>(110 lb)          |

\*380 mm (15 inch) without seat and rear wheels

\*\*with solid wheels

\*\*\*frame without rear wheels

## Frame size 1 & 2

|                                                                                   |                                                 | mm (inch), kg (lb)                                                                       |                       |                                                                                          |                       |
|-----------------------------------------------------------------------------------|-------------------------------------------------|------------------------------------------------------------------------------------------|-----------------------|------------------------------------------------------------------------------------------|-----------------------|
|                                                                                   |                                                 | Size 1  |                       | Size 2  |                       |
|                                                                                   |                                                 | Normal                                                                                   | Suspension            | Normal                                                                                   | Suspension            |
|  | Mass of the heaviest part***                    | 7.2 kg<br>(15.9 lb)                                                                      | 8.2 kg<br>(18.1 lb)   | 7.4 kg<br>(16.3 lb)                                                                      | 8.8 kg<br>(19.4 lb)   |
|  | Pivot width                                     | 1160 mm<br>(45¾ inch)                                                                    | 1160 mm<br>(45¾ inch) | 1200 mm<br>(47¼ inch)                                                                    | 1200 mm<br>(47¼ inch) |
|  | Maximum slope<br>(for use of<br>driving brakes) | 7°                                                                                       | 7°                    | 7°                                                                                       | 7°                    |
|  | Static stability<br>Sideways                    | 10°                                                                                      | 10°                   | 10°                                                                                      | 10°                   |
|  | Static stability<br>downhill                    | 10°                                                                                      | 10°                   | 10°                                                                                      | 10°                   |
|  | Static stability<br>uphill                      | 10°                                                                                      | 10°                   | 10°                                                                                      | 10°                   |

\*380 mm (15 inch) without seat and rear wheels

\*\*with solid wheels

\*\*\*frame without rear wheels

mm  
(inch)  
kg  
(lb)

## Seat size 1 & 2

|                                                                                                                                              | mm (inch), kg (lb)                                                                       |                                                                                          |
|----------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------|
|                                                                                                                                              | Size 1  | Size 2  |
|  Backrest recline angle                                     | 80-160°                                                                                  | 80-160°                                                                                  |
|  Backrest height with head support plate                    | 505-665 mm<br>(20-26¼ inch)                                                              | 620-810 mm<br>(24½-32 inch)                                                              |
|  Backrest height without head support plate                 | 365-525 mm<br>(14½-20¾ inch)                                                             | 420-610 mm<br>(16½-24 inch)                                                              |
|  Back width, top of backrest (with side supports)           | 150-225 mm<br>(6-8¾ inch)                                                                | 170-260 mm<br>(6¾-10¼ inch)                                                              |
|  Back width, top of backrest (without side supports)        | 315 mm<br>(12½ inch)                                                                     | 365 mm<br>(14½ inch)                                                                     |
|  Effective seat depth                                      | 180-340 mm<br>(7-13½ inch)                                                               | 240-420 mm<br>(9½-16½ inch)                                                              |
|  Effective seat width (without hip supports)              | 330 mm<br>(13 inch)                                                                      | 380 mm<br>(15 inch)                                                                      |
|  Effective seat width minimum/maximum (with hip supports) | 180-260 mm<br>(7-10¾ inch)                                                               | 230-310 mm<br>(9-12¼ inch)                                                               |
|  Foot support to seat                                     | 130-260 mm<br>(5-10¼ inch)                                                               | 185-420 mm<br>(7¼-16½ inch)                                                              |

\*seat only (without accessories)

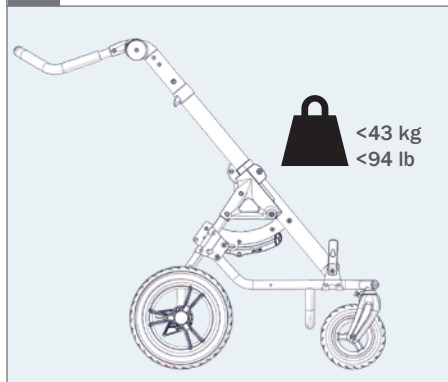
## Seat size 1 & 2

|                                                                                                                         | mm (inch), kg (lb)                                                                       |                                                                                          |
|-------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------|
|                                                                                                                         | Size 1  | Size 2  |
|  Folded height                         | 275 mm<br>(10¾ inch)                                                                     | 295 mm<br>(11½ inch)                                                                     |
|  Folded length                         | 535 mm<br>(21 inch)                                                                      | 635 mm<br>(25 inch)                                                                      |
|  Folded width                          | 380 mm<br>(15 inch)                                                                      | 430 mm<br>(17 inch)                                                                      |
|  Mass device*                          | 5.3 kg<br>(11.7 lb)                                                                      | 6.3 kg<br>(13.8 lb)                                                                      |
|  Maximum rated load                    | 42.3 kg<br>(93.2 lb)                                                                     | 58.3 kg<br>(128.5 lb)                                                                    |
|  Maximum user mass                     | 35 kg<br>(77 lb)                                                                         | 50 kg<br>(110 lb)                                                                        |
|  Maximum user mass in transportation  | 35 kg<br>(77 lb)                                                                         | 50 kg<br>(110 lb)                                                                        |
|  Minimum user mass in transportation | 22 kg<br>(48 lb)                                                                         | 22 kg<br>(48 lb)                                                                         |
|  Seat recline angle                  | (-5°)-(40°)                                                                              | (-5°)-(40°)                                                                              |


\*seat only (without accessories)



## Size 1


## Size 2

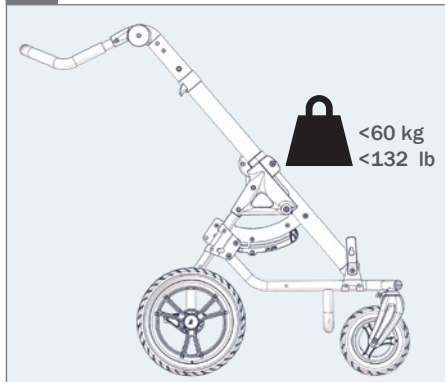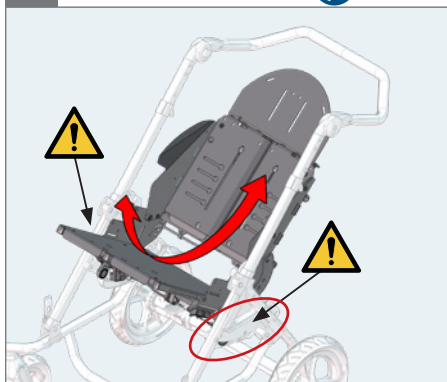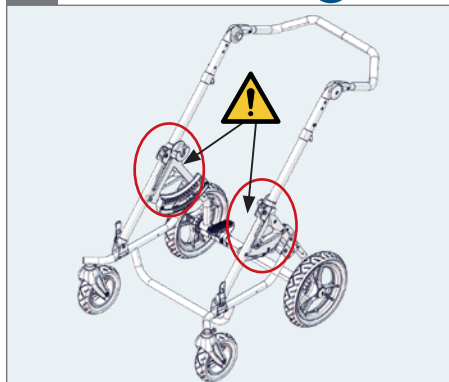
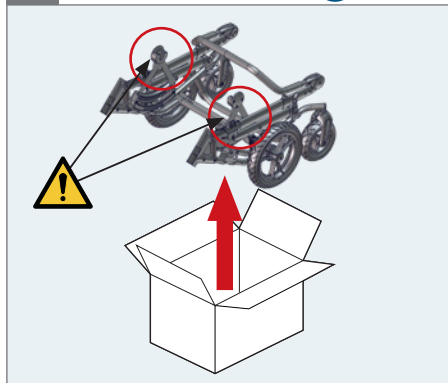
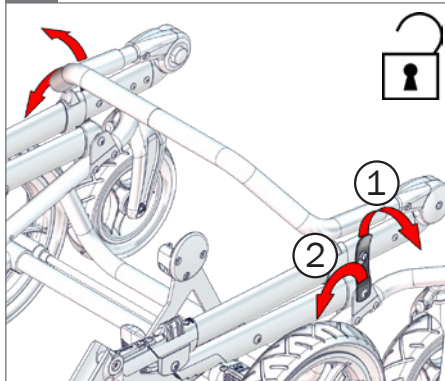
1.

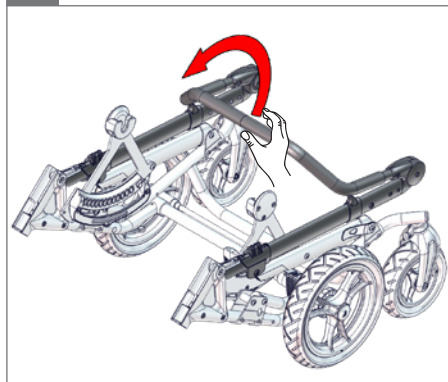
2.

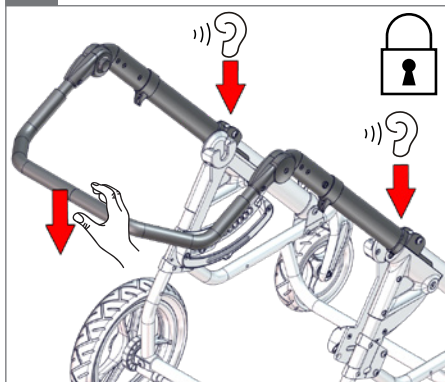3.



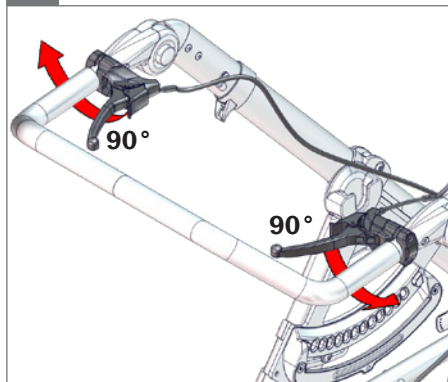
1.







2.

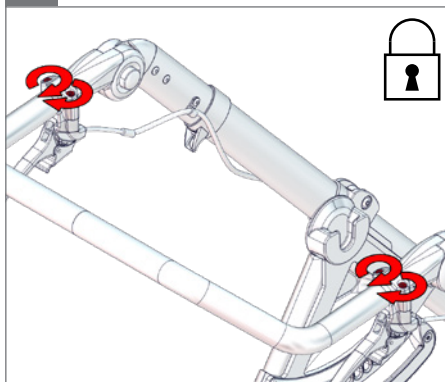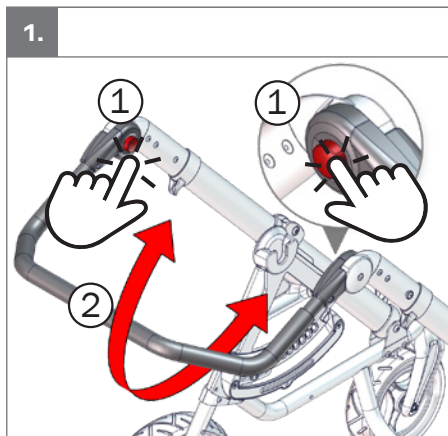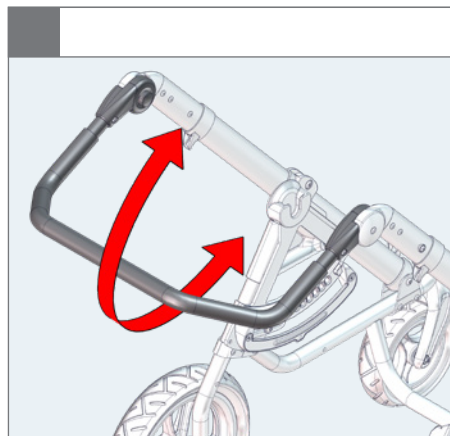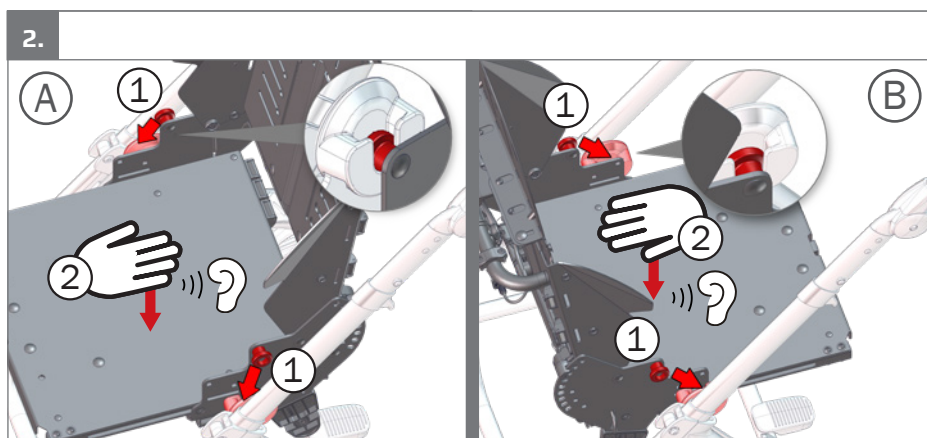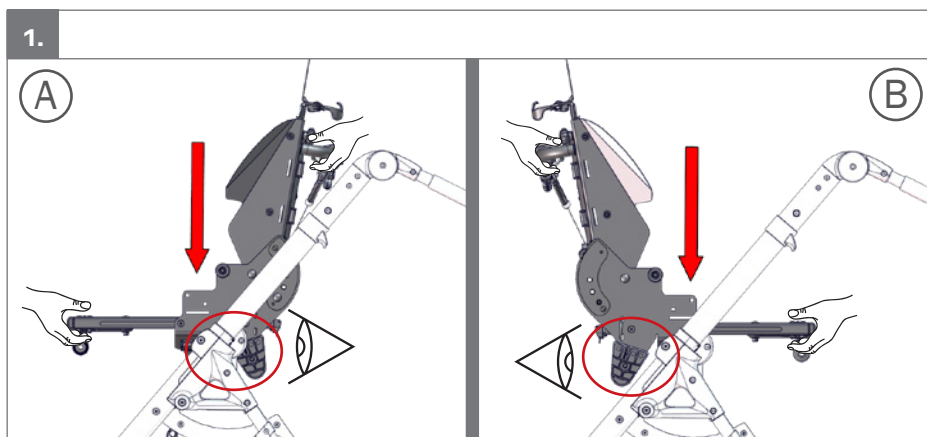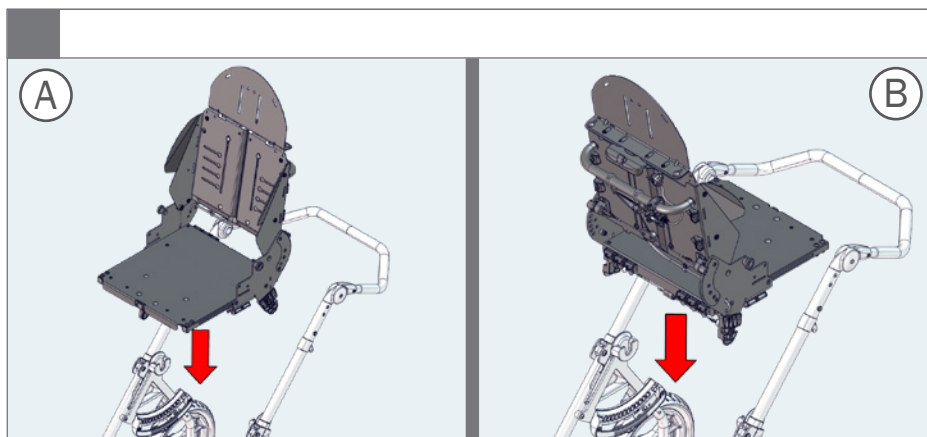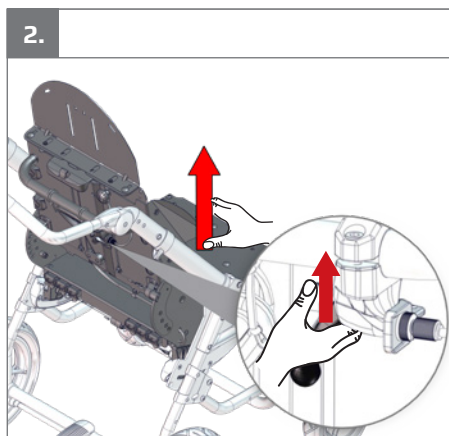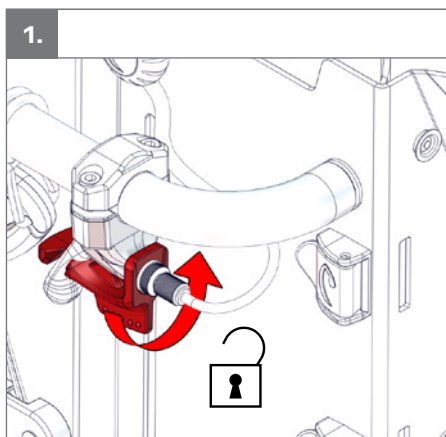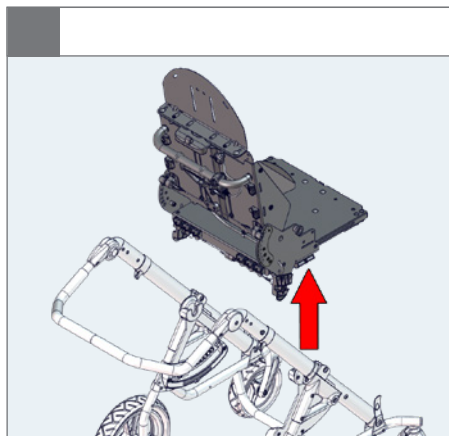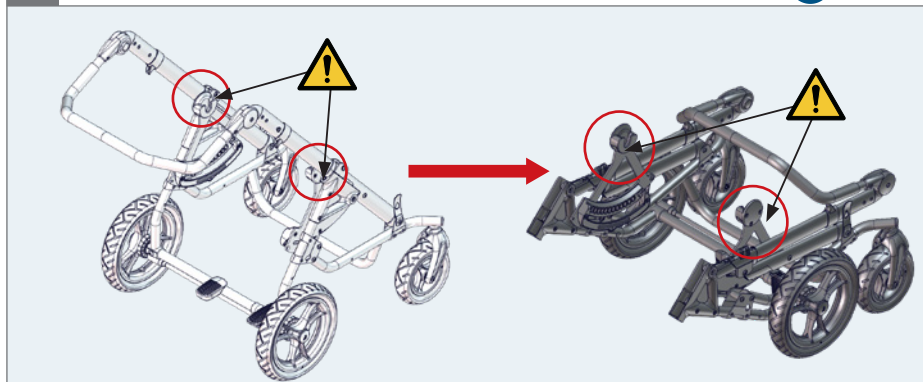
4 mm


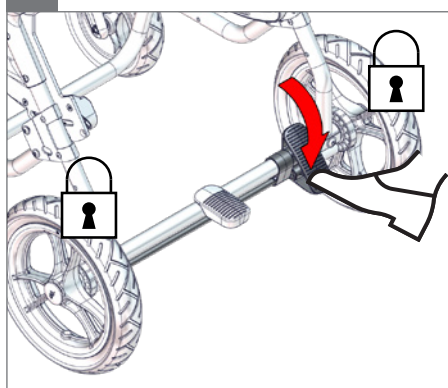
1.

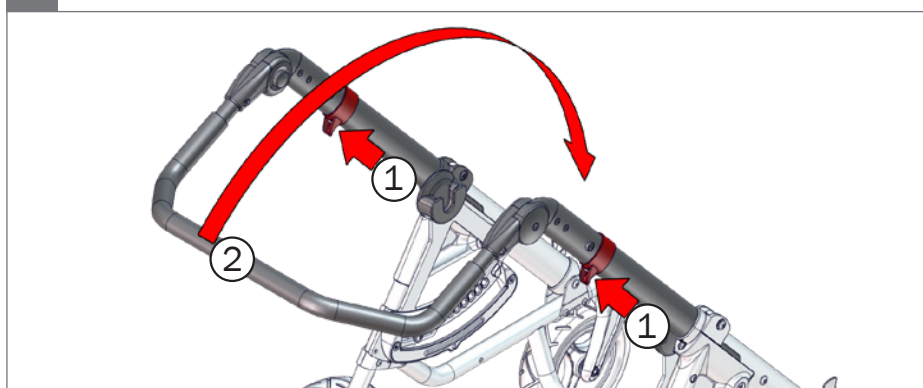
2.



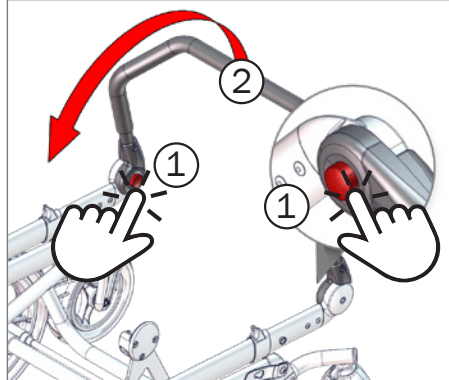
3.

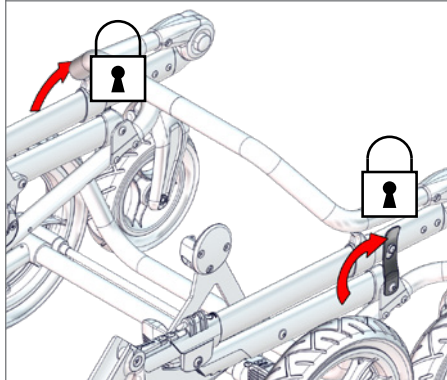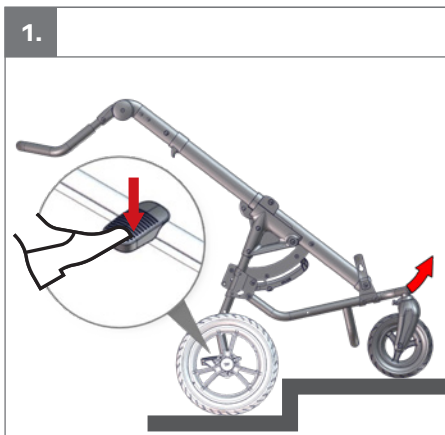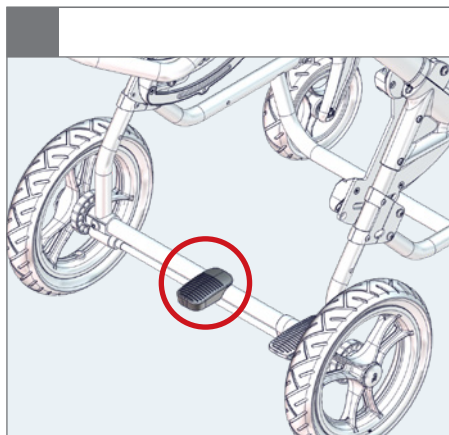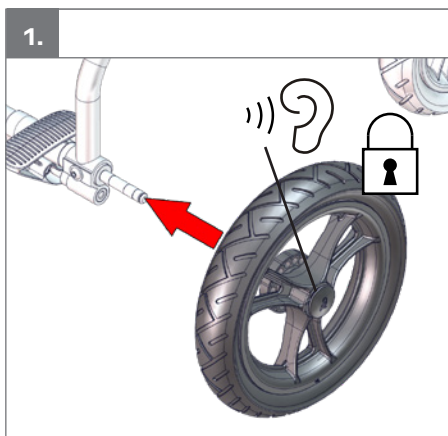
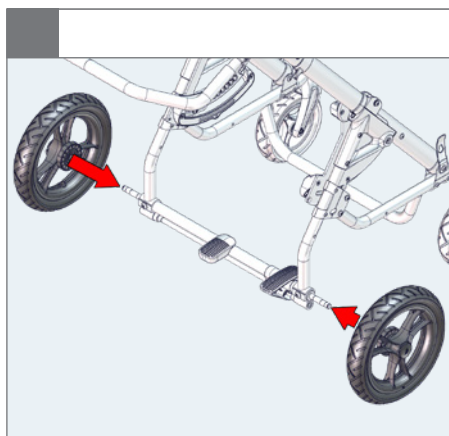
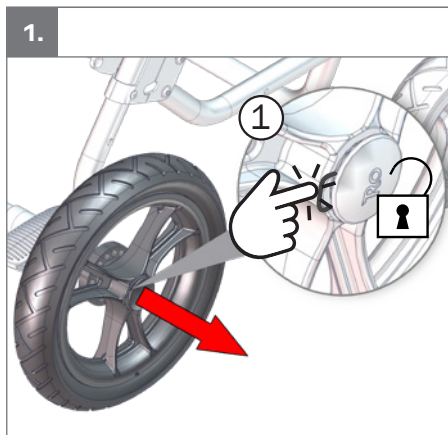
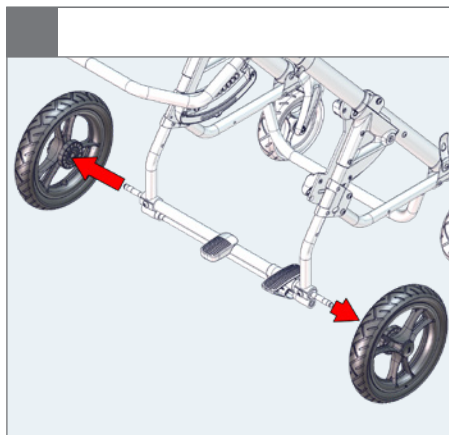
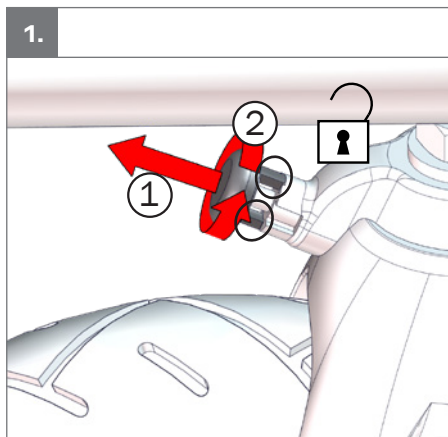
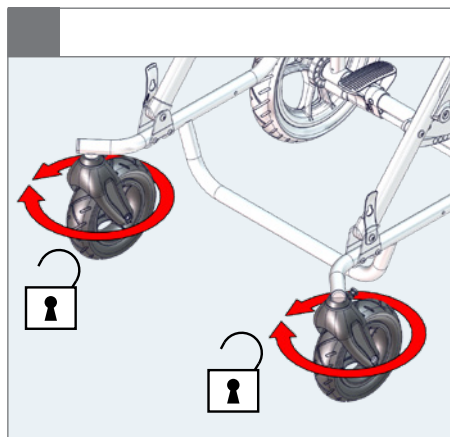
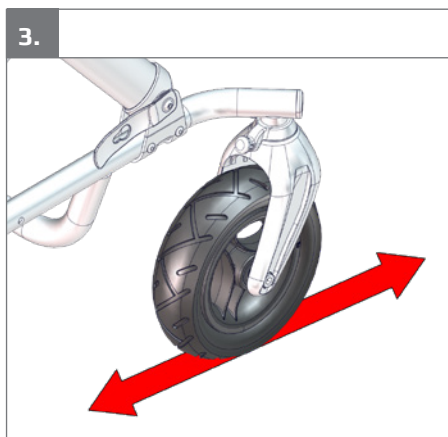
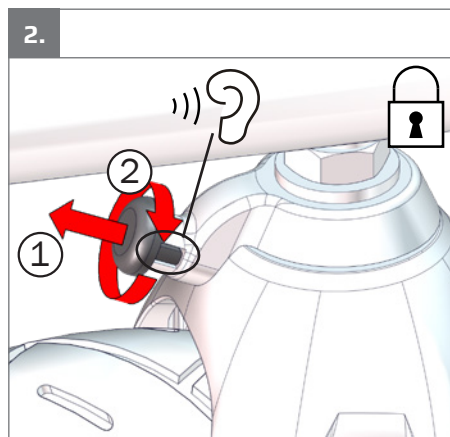
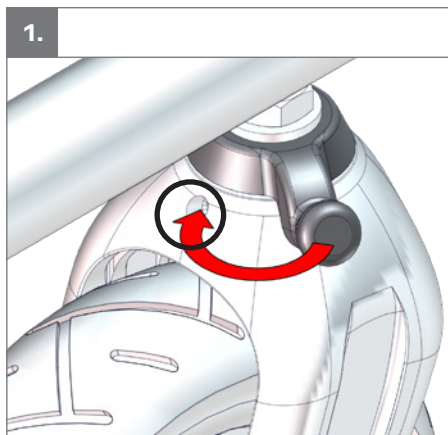
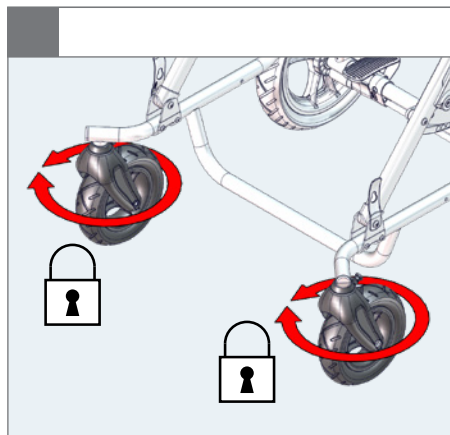
4.

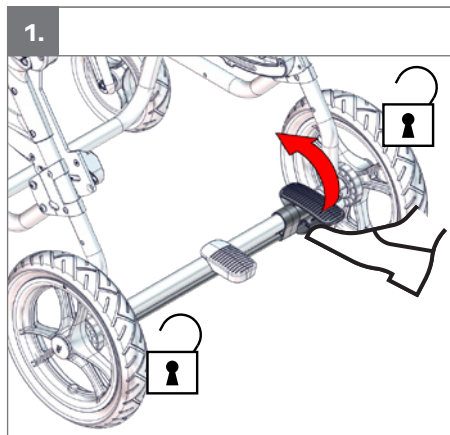
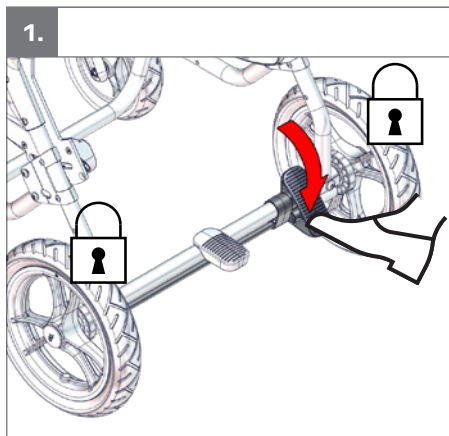
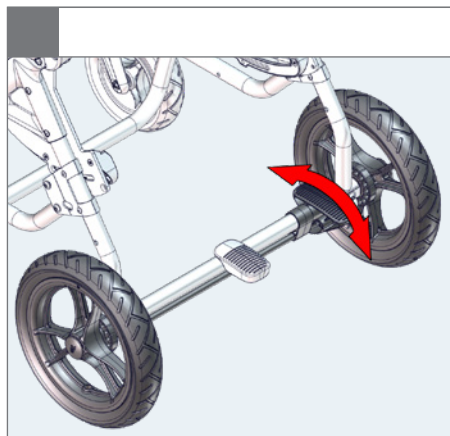
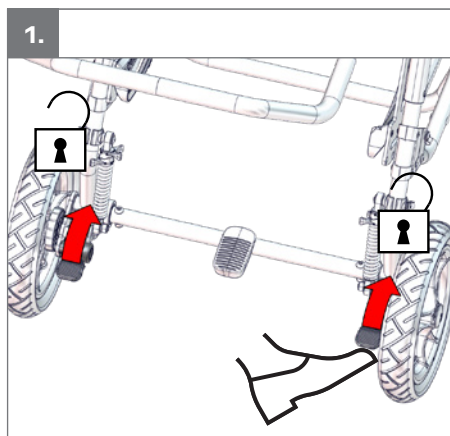
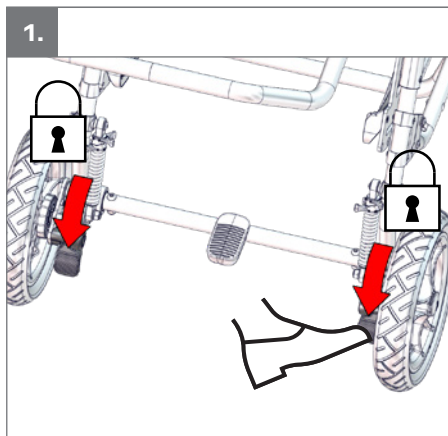
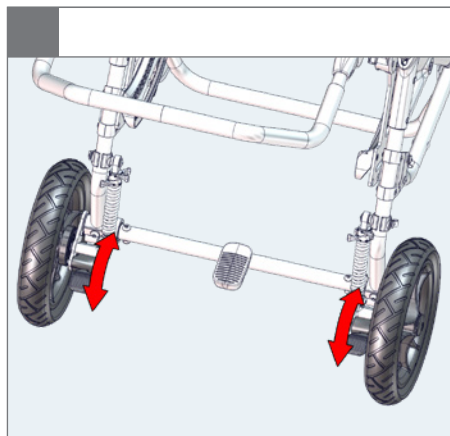
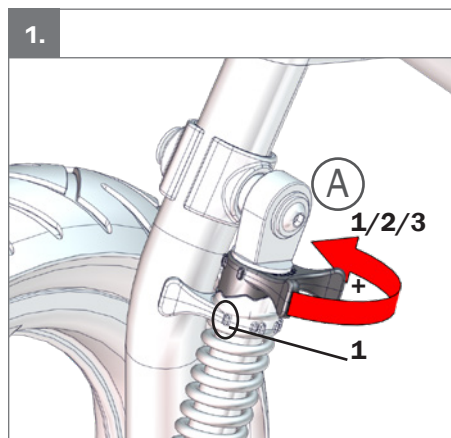
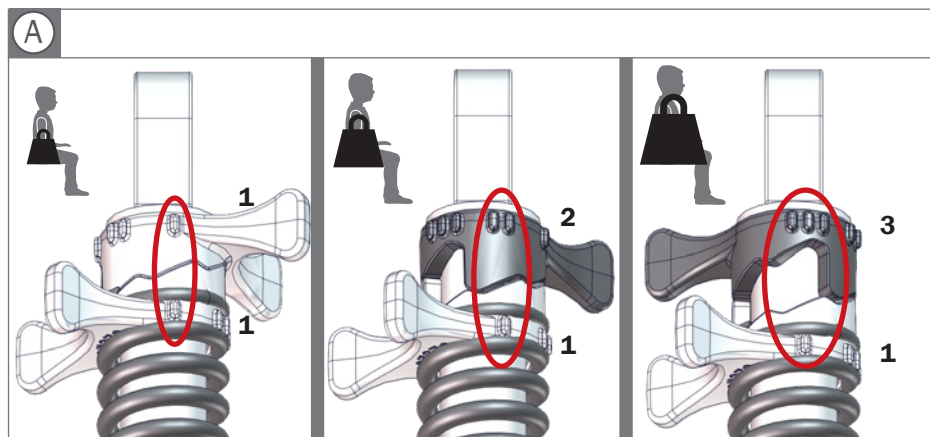
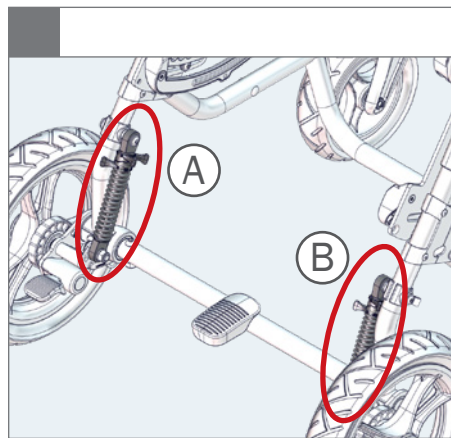
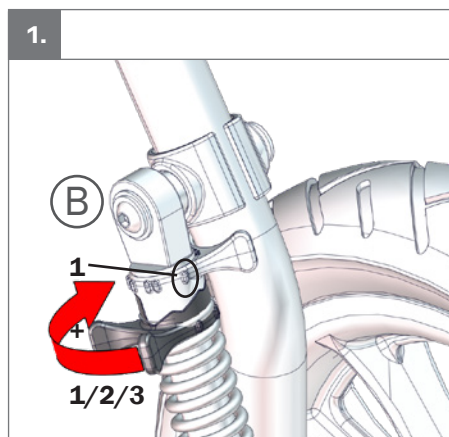
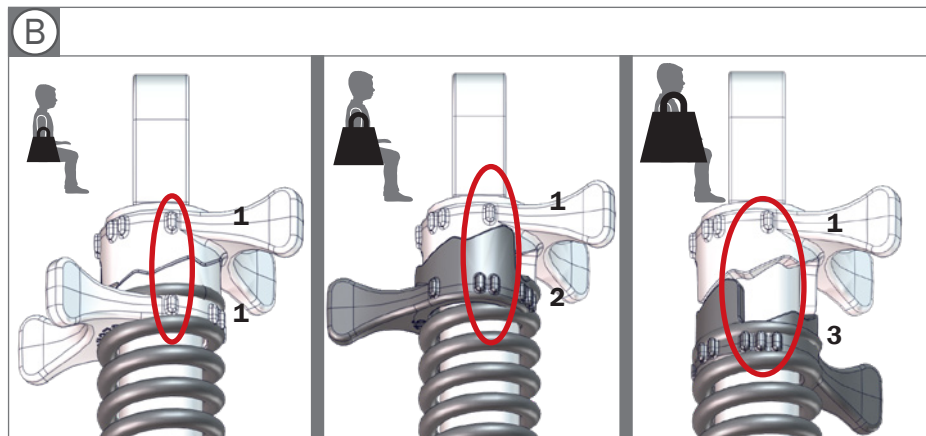
2.







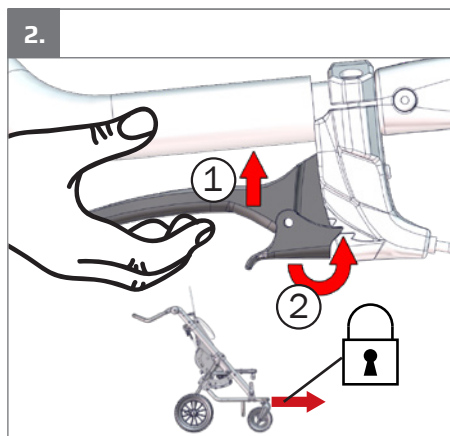
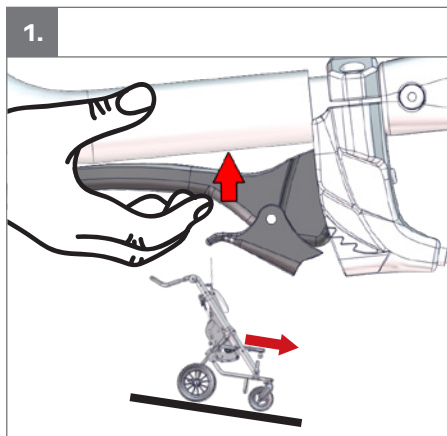
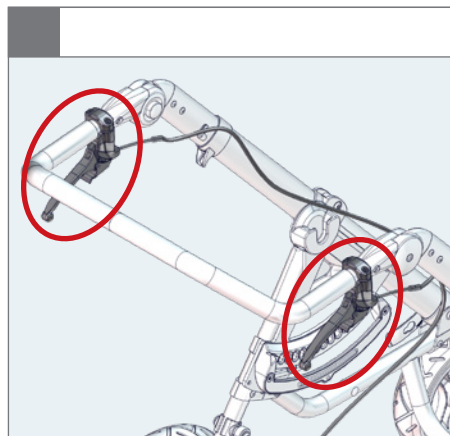
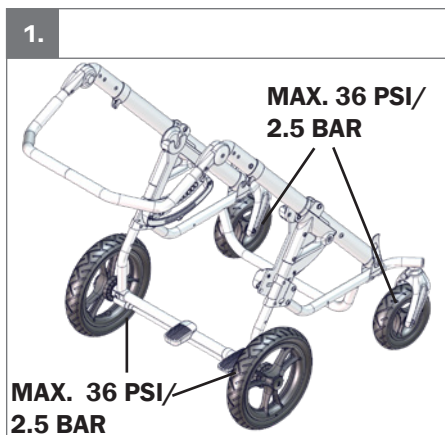
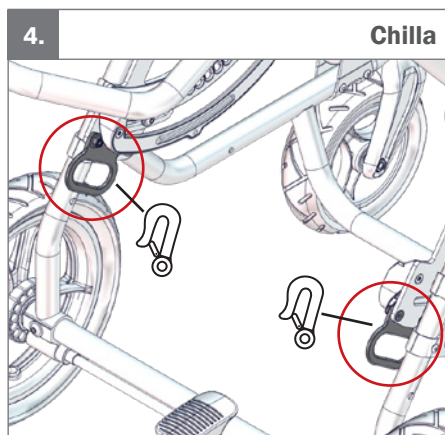
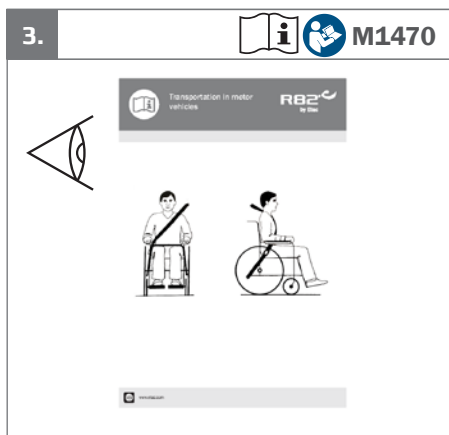
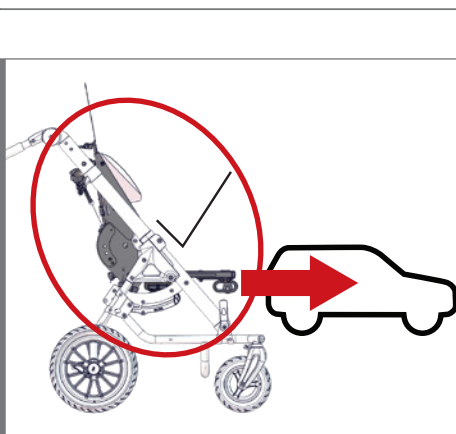
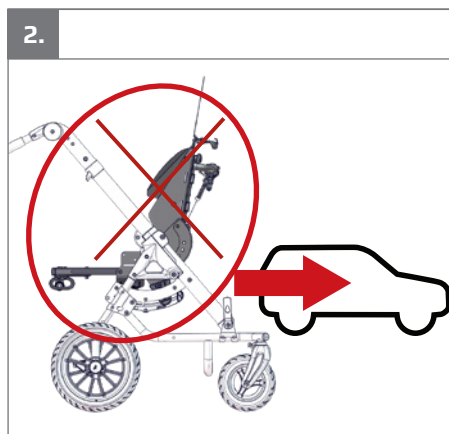
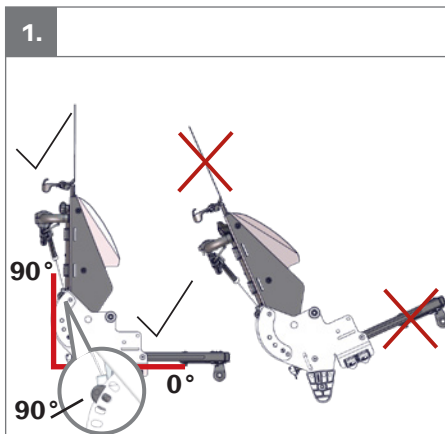
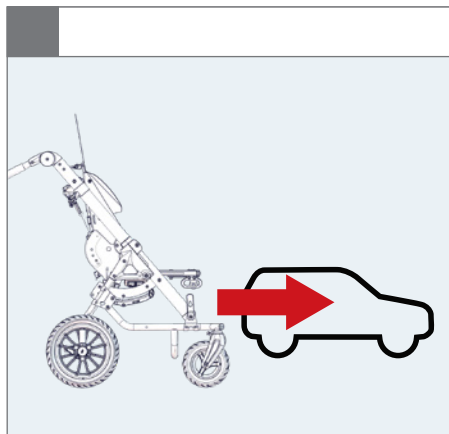
5.

## Chilla seat


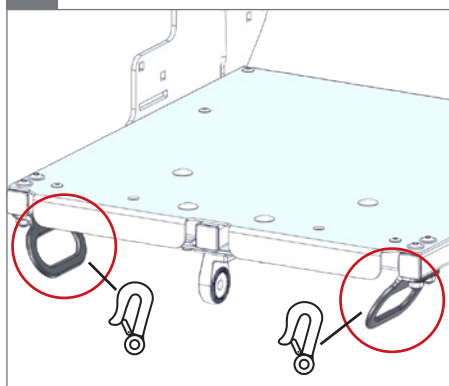
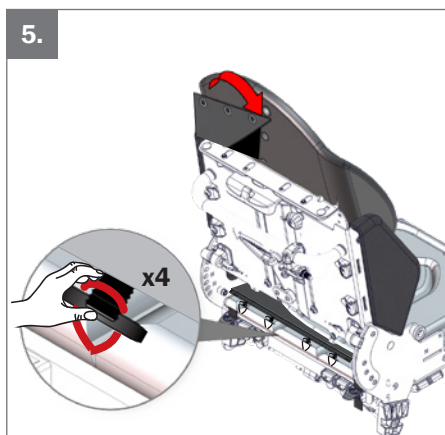
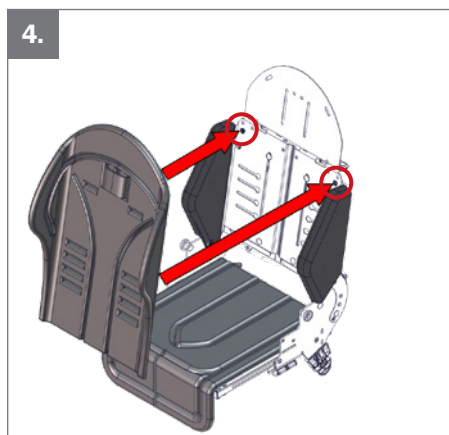
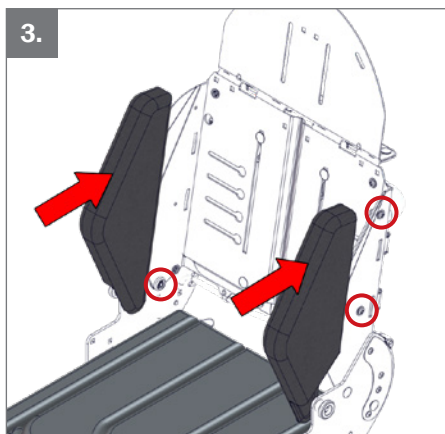
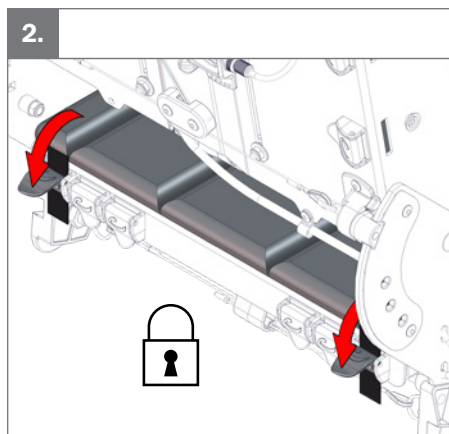
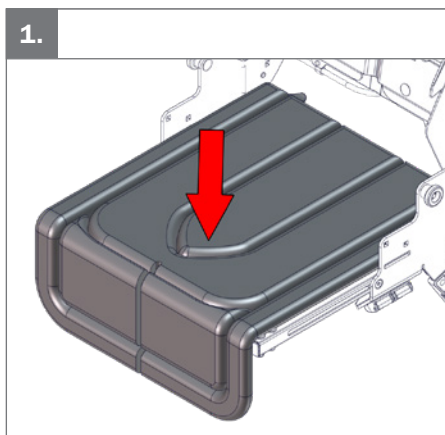
6.

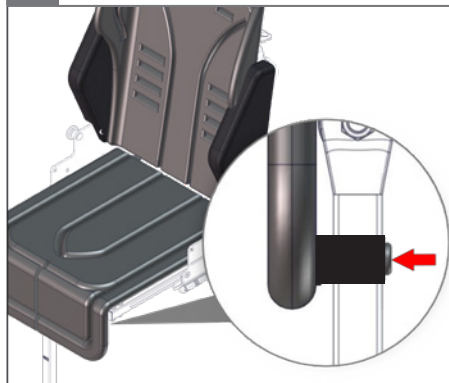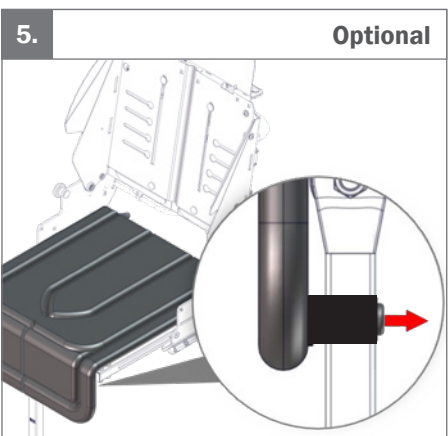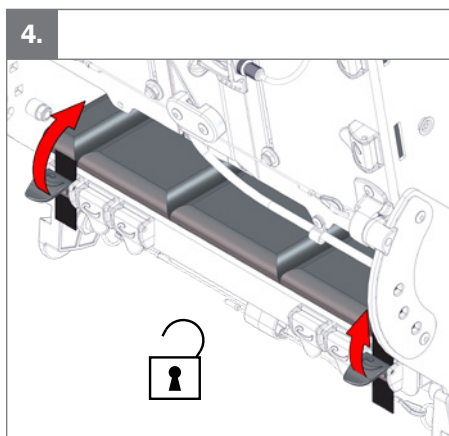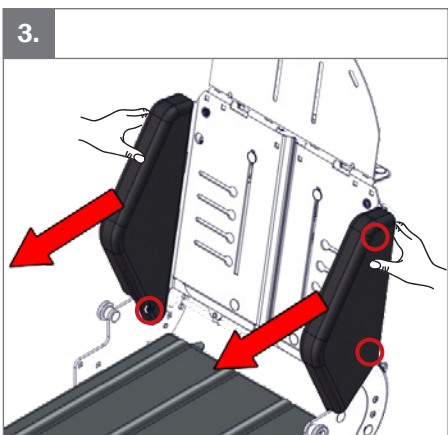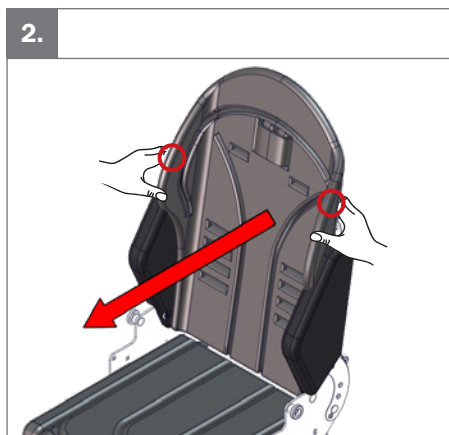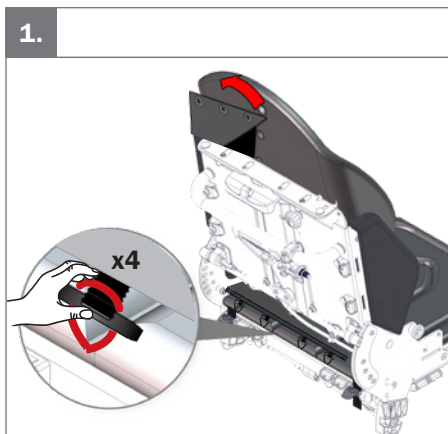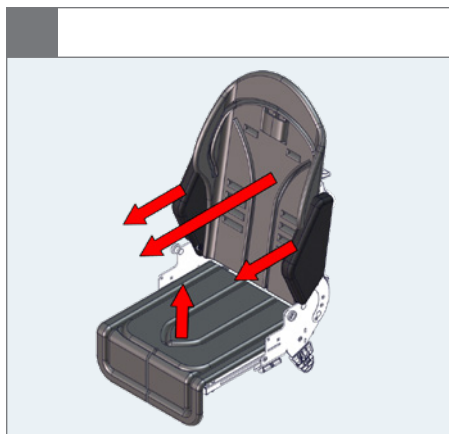
Optional




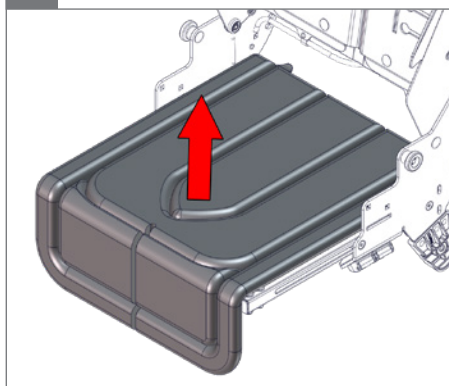
6.

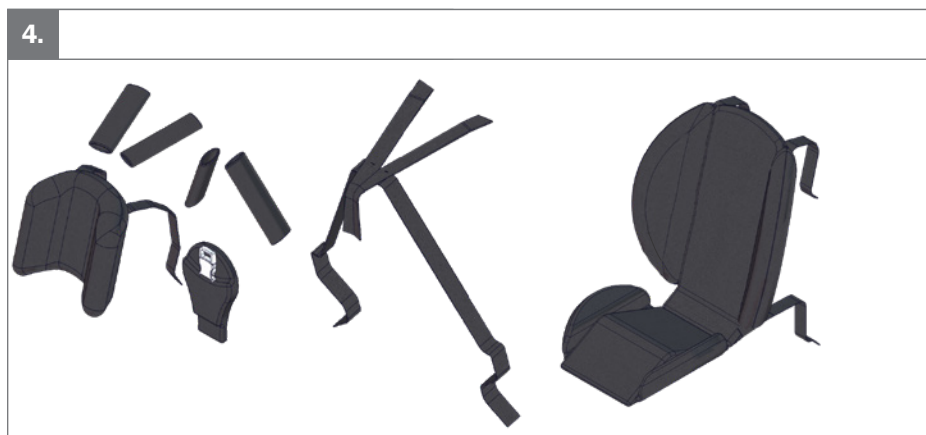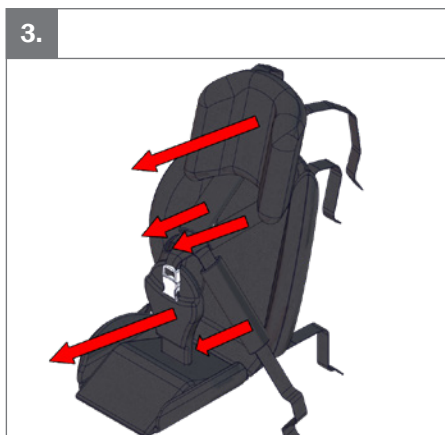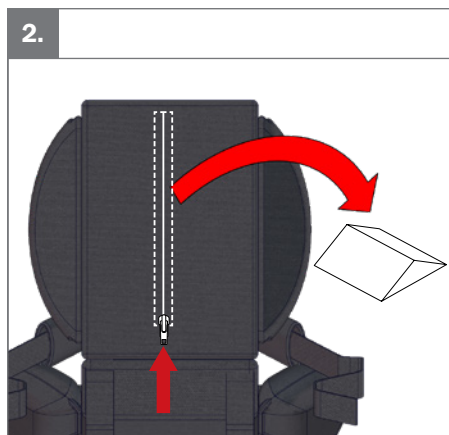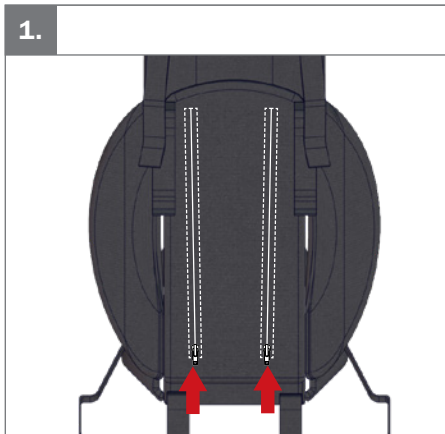7.




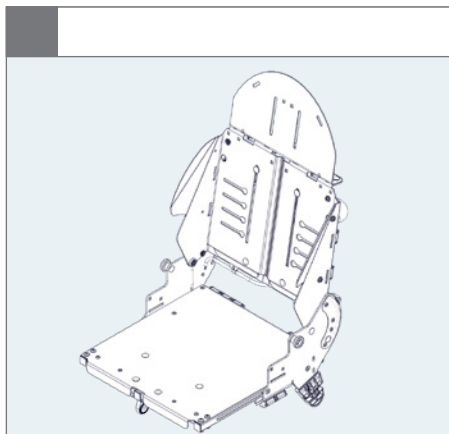
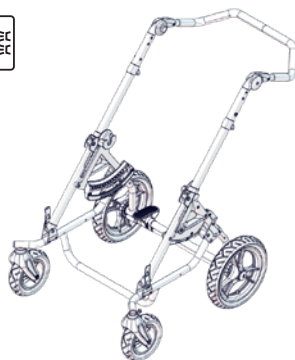
1.

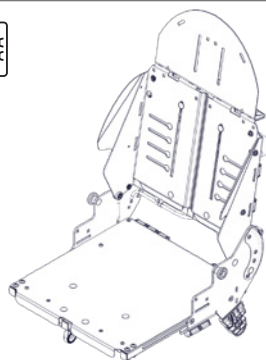
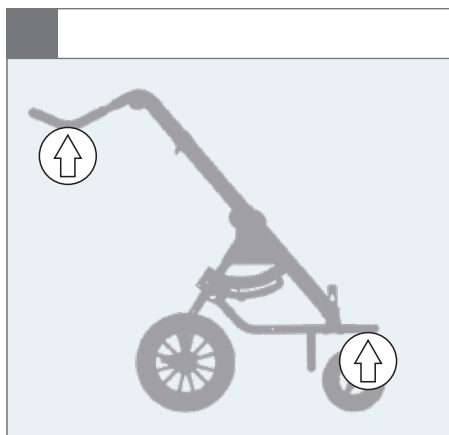
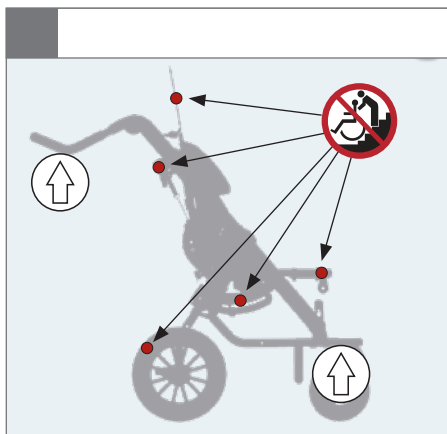
1.

### Chilla seat

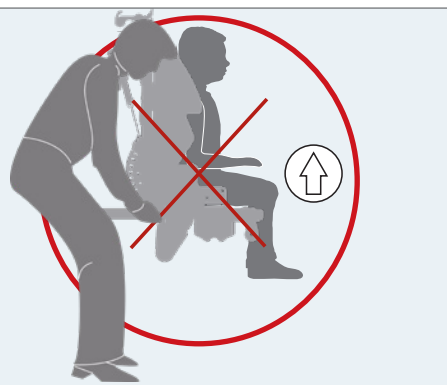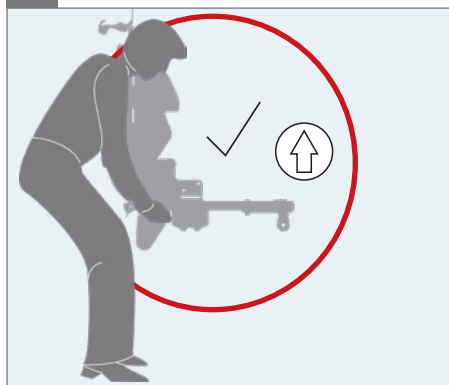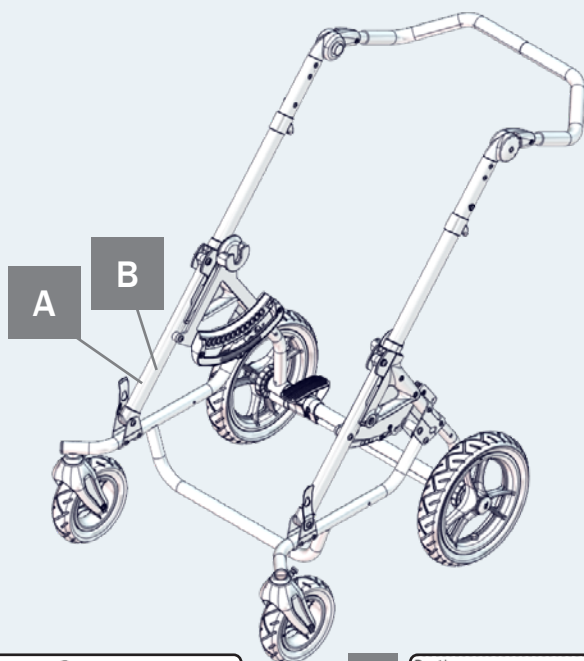A

**R82**  
by Etac

R82 A/S, Parallelvej 3  
DK-8751 Gedved  
etac.com  
Made in xxxxxxxx

MD

CE

UK  
CA

YYYY-MM-DD

B

Posting group  
Product name, size x

DMR ver.: xxxxx

Max. load: xxx kg  
Max. load: xxx kg  
Max. rated load: xxx kg  
Mass device: xxx kg

xxxxxx

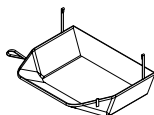
xxxxxx

xxxxxx


xxxxxx

9996098191

9996098170

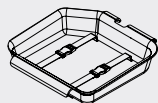9996098171

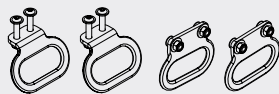
9996098173


9996098174

9996098175

9996098169

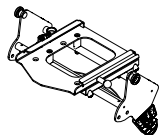9996098193

9996098203

9996098207

## General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this user manual before use and save it for future reference.

### Intended purpose

Chilla is an assistive device intended for compensation for a functional impairment due to an injury or a disability. Chilla is designed for an individual lacking the ability to maintain a seated position or move around without the use of an assistive device due to mobility impairments.

### Intended user group

The target group for the device constitutes children and adults with a need for sitting transportation due to impaired mobility and functional ability. The size of the device is dependent on weight and physical dimensions of the user.

Secondary users of the device are caregivers providing assistance to the primary user as well as clinicians/technicians who set up the device.

### Indications

Indications for use are a need for sitting transportation due to persistent, significant, or substantial impairment of mobility, i.e., disability including, but not limited to, cerebral palsy, muscular dystrophy, neuromuscular diseases, different syndromes, structural or functional lower limb injuries, or an impaired ability to uphold postural control and balance.

### Application of the device

The user is the person sitting in the device. The caregiver is the person operating the device.

The exposure will be limited ( $\leq 24$  h). The user's need for an assistive device for seating can be short term ( $\leq 30$  days) or long term ( $> 30$  days). The device will come into contact with intact skin. Depending on the configuration and the accessories used the contact will often be with the skin of the upper extremities and lower extremities of the user and caregiver.

The use of the device will be discontinued multiple times during the day when the user will be lying down, standing or sitting in another assistive device or a conventional product.

### Contraindications

There are no known contraindications.

### Precautions

Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

Special attention should be given to the considerations about the configuration of the device and the use of accessories for users with limitation in their:

- sensory functions and the sensation of pain: Consider extra checks of the pressure caused by the support provided by the device and its accessories.

- movement function (e.g. experience involuntary movements): Consider using a device with a dynamic backrest.
- ability to maintain a seated position: Use the device with accessories (e.g. side supports, vests, belts or arm supports and a tray).
- ability to maintain the position of their head: Consider using accessories that provide support to the positioning of the head.

### Application areas

Chilla is intended for indoor and outdoor use, in home environments or institutions but is not intended for wet areas.

### Enclosed tools:

- N/A

## CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745).

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings.

### Combinations

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

### Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

### Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for

further information.

## **R82 Warranty**

[www.etac.com/support](http://www.etac.com/support)

### **Presales information and instructions**

Presales information and instructions are available at [www.etac.com](http://www.etac.com), under your product, in the documents section.

## Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

### Caregiver

- Read the user manual carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user.
- Always use the correct lifting techniques and aids for this purpose.
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver.
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier.
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user manual.
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible.
- When adjusting the seat angle, keep a hand on the seat mounted on the frame to ensure slow angling.
- Ensure that moving parts do not squeeze or trap body parts when adjusting them.
- Ensure that body parts are not trapped in openings for mounting of supports/accessories.
- Always use the product in a moderate pace. Do not run with the product.
- Information about transportation can be found in the document "M1470 Transportation in Motor Vehicles" supplied with the product.
- Information about roadability can be found in the document "M1450 Roadability" supplied with the product.
- Information regarding frame and seat combinations including max. load in combination can be found at: etac.com.
- This product contains small parts which could present a choking hazard if removed from their intended placing.

### Environment

- Observe the surface temperature on the product, before placing a user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use.
- Use this product on a level surface without

any obstacles. Be aware of reduced maneuverability on slippery surfaces.

- When using on inclines, please refer to measurements.

### User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered.
- Do not place more than one user in the product.
- When changing user, clean the product and disinfect it if necessary. Carry out necessary adjustments after cleaning.

### Product

- Removable parts without the use of a tool: Rear wheels and most accessories. The removal will have an adverse effect on the device.
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children.
- Activate the brakes before placing the user into the product.
- Ensure the wheels are fully functional and secure before use.
- The correct tyre pressure is printed on the tyre casing and listed in the section for tyre technical data.
- In case of a tyre puncture do not use the product until remedied.
- Ensure the stability of the product is maintained before placing the user in it.
- When the footplate is close to the floor and the castors, the castors may be unable to swivel.
- Inspect the product and all of its accessories and replace any worn out parts before use.
- Do not expose the gas damper to pressure or high temperatures. DO NOT puncture.
- The product is approved for transportation in vehicles according to ISO 7176-19. More information can be found in the document "M1470 Transportation in Motor Vehicles" supplied with the product.
- In accordance with ISO 7176-19 annex D the product is evaluated regarding the overall rating of belt restraint accommodation. Rating for: ease of properly positioning a three

point belt restraint.

The product receives a score of "excellent".

Rating for: the degree to which proper three point belt restraint positioning can be achieved.

The product receives a score of "Good".

- The device conforms to ISO 7176-8, covering requirements and test methods for static, impact and fatigue strengths.
- Unoccupied device is suitable for land and air transport.
- Depending on the size, the device is intended for occupant mass group I (ISO 7176-5).
- This device is tested for resistance to ignition at level non-ignition in accordance with applicable standards.
- The device might lose its flame resistant characteristics if cushions other than original R82 are used.

## Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device.

Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the device. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

### Lifetime

The lifetime of this device in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

### Reuse

This device and its accessories/components are suitable for reuse.

### Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

### Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the device is inspected yearly, every time the device is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the device.

### Service checklist:

Check and adjust the following as per their intended use:

- Cushioning.
- The push-brace can be angle adjusted.
- If equipped with airfilled tyres, check the air pressure.
- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Wheels are moving freely and that all wheel locks/brakes are easily activated.
- Nuts and bolts on the product are tightened.
- Fittings and buckles for signs of worn out parts.
- Product for cracks or signs for worn-out parts.
- Seat docking is clean and functional.
- Product can be folded for storage or transportation.
- Seat back is adjustable and can be locked in position.
- All labels on the product are intact.

Repair or replace any damaged or worn out parts.

### Maintenance

Before every use it is recommended to:

Wipe off any residue and dirt from the device, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use.

Carry out a visual inspection, for any damaged or worn-out parts.

### Washing

#### Main device

This device can be washed at 60 °C using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the device. Unmount all accessories and wash them separately.

Tilt/rotate the device after washing to facilitate water drainage from within the device.

### Fabric and covers

The cover can be washed at max. 60 °C using a mild detergent in a washing machine.

### Foam

It is not recommended to wash the foam parts.

### Handwash

This device and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

### Disinfection

The device can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the device, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

### Materials

- Aluminium
- Plastic
- Rubber
- Polyurethane
- Neoprene
- Ipanema
- Steel
- Zinc alloy.

### Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized.

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**Etac Ltd.**
**Gelbart AG**