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User Guide - UK (19-34)

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Instrukce - CZ (67-82)

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Stingray Swing-out

2020-11

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Gebruiksaanwijzing NL	3 - 18
User Guide UK	19 - 34
Gebrauchsanweisung DE	36 - 50
Istruzioni IT	51 - 66
Instrukce CZ	67 - 82
Mode d' emploi FR	83 - 98

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DECLARATION OF CONFORMITY: MANUFACTURER

Company name	:	Peereboom BV
Address	:	Spieringweg 931
Postal code	:	2136 LK
Place	:	Zwaanshoek
Country	:	The Netherlands
Phone	:	0031 (0)23 561 73 19
Fax	:	0031 (0)23 563 94 99
E-mail	:	Info@Peereboom.com
Website	:	www.Peereboom.com

DESCRIPTION AND IDENTIFICATION OF THE PRODUCT

Name	:	Swing-Out
Brand	:	Stingray Swing-out size 1/2/3/4
Function	:	Car seat for children with a body weight of 9 to 36 kg
Serial number	:	xxxx

COMPLIANCE

The manufacturer declares that the above mentioned product fulfills all relevant provisions of

Medical Devices Regulation 2017 / 745

for the design and manufacture

NEN-EN ISO 9001:2015; NEN-EN ISO 14001:2015;
EN 12182:2012; EN-ISO 14971:2012; ECE R44/04;
Flammability acc. to EN 71-2; Toxicity acc. to EN 71-3

Place: Zwaanshoek
The Netherlands
Dated: 8 April 2020

Name: H.A. Peereboom
Function: Company Director
Signature: 

User Guide UK

TABLE OF CONTENTS

Stingray Swing-out	20
Isofix	21
Assembly	22
Important information	22
Safety	23
Guarantee	24
Maintenance	24
Fitting the Stingray Swing-out	
with Easyfit mounting system	25
with Evo tensioning belt (NL)	26
Using Stingray Swing-out	27
Adjusting seat depth	28
Seat width	28
Tilt adjustment	28
Footplate	28
Adjustable pelvic support	28
Side impact headrest	29
Anatomical neck cushion	29
Side supports	29
Outer knee supports	29
Rear seat protection	29
5-point harness Bellypad	30
Product identification labels	32
Measurements	33
Manufacturer/distributor details	33
Declaration of Conformity According to 94/42/EEG, Class 1	34

STINGRAY SWING-OUT

The Stingray Swing-out is a car seat for children weighing from 9 to 36 kg. It comes in 4 sizes and the Easyfit frame turns and partially slides in and out of the car. This makes it easier for carers to put children with a disability in and out of the car. The seat depth and side impact headrest are adjustable to allow for the growth of the child. Furthermore, several accessories are available that can give the child better support.

The Stingray Swing-out is available with two different mounting systems:

- Easyfit mounting system for 4/5-door cars with the Isofix anchoring system in the rear seat (usually outer two seats).
- or
- Evo-tensioning belt restraint system for 4/5-door cars without Isofix anchoring system or for use in the front passenger seat (NL).

Both versions are available in a right or left-hand version. The Stingray Swing-out cannot be converted from right to left.

The Stingray Swing-out has been approved according to the ECE 44.04 standard and the headrest has been tested "Side Impact" according to the ISO/DIS14646 standard

For information: most of the Stingray buggy accessories are not suitable for the Stingray Swing-out. This is to ensure compliance with ECE 44.04.

ISOFIX

Isofix is a modern solution for placing a child's seat in the rear seat of a car in a safe and easy way with standardized anchor points. For car seat groups 2 and 3, the Isofix anchor points are an additional fastener for the car seat and therefore the car's 3-point seat belt (approved according to ECE R16) remains mandatory. Over the years, Isofix has become the international standard for car manufacturers and car seat manufacturers. This anchoring technique minimises fitting mistakes and optimises the protective effectiveness through a strong connection between child seat and car.

The Stingray Swing-out with the Easyfit mounting system (A) uses the car's Isofix anchor points. The child in the Stingray Swing-out must always continue to use the car's 3-point safety belt (approved in accordance with ECE R16). By using the Isofix anchorage, the Stingray Swing-out stays in place as closely as possible during an accident, including side impacts. It also ensures optimum power distribution through the stable mounting of the car seat with the car. The adjustable rod (B) of the Easyfit frame ensures even better stability of the frame, even during an accident. Use must always be made of the (correctly attached) three-point belt (approved according to ECE R16) and the Side-Impact headrest. In a number of cases, the Isofix brackets can be installed in the car afterwards. Ask your car dealer about this.

According to European legislation, ISOFIX will be mandatory in all new models with the exception of two-seater cars, launched from November 2012, and in all vehicles manufactured after November 2014.

ASSEMBLY

It is recommended that an approved fitter installs the car seat into the car. This could be your supplier or a car dealer. Then parents/carers will also be instructed how the Stingray Swing-out is fitted in the car and how to remove it. To prevent incorrect fitting, these instructions must be followed step by step. The instruction labels on the product must always be followed.

Incorrect assembly can create a dangerous situation and endanger lives. If there are doubts about the installation or use in the car, you should contact your supplier or manufacturer.

Due to the shape of the frame and method of mounting, an impression may remain in the seat cover over time. To prevent this, you can use the R82 positioning or protective cushions (item no. NLSW070 / 071, available via your supplier) on your rear seat. Make sure that the car seat can slide out freely and does not rub on the cheek of the upholstery. Trespa protection plates are available for this (43x14 cm, article number NLPE011).

IMPORTANT

- It is important to choose the right size of Stingray Swing-out car seat for your child. If you have any doubts, you should contact your supplier or the manufacturer.
- Make sure that the car's three-point seat belt (approved according to ECE R16 for EU and UK road use) is always around the Stingray Swing-out (even without a child in it) so that the central bolt of the Stingray Swing-out seat cannot break off in the event of an accident and become loose in the car. The Easyfit system or the Evo tensioning belt is not sufficient to hold the Stingray Swing-out in place, it is more than 25 kg.
- It is not permissible under ECE Regulations to make modifications to this car seat.
- Never leave your child unattended in the car.
- Position the three-point belt correctly over the shoulder and over the hips and ensure that it does not run over the child's stomach but over the pelvis.
- Make sure that the belt runs flat over the child's body.
- The Stingray Swing-out must be turned back and locked during transport and must therefore be fitted with the car's 3-point safety belt.
- Metal parts of the car seat can be heated by the sun. Ensure that the skin does not come into direct contact with these parts because of possible blistering.
- Make sure that delicate parts of the Stingray Swing-out are not damaged during use. If the car seat does not function properly, have it checked or replaced.

- After an accident, this car seat can be damaged, although this is not always visible. Have this checked and, if necessary, replace the car seat (frame, seat unit and Easyfit mounting system).
- Under ECE Regulations the Stingray Swing-out may only be fitted with its original accessories and parts.
- A limited number of accessories are available including a 5-point belt, side supports for the trunk, pelvic supports and a footrest. Most accessories for the Stingray buggy are not applicable with the Stingray Swing-out. Please contact your supplier for further information.
- During transport, the luggage must always be properly stored and secured. In the event of an accident, this can fly through the car and cause serious personal injury.
- It should be pointed out to the user/parent that the fixed and plastic parts of the car seat are positioned and mounted in such a way that they cannot get caught in a movable seat or in a door of the vehicle during daily use.
- Make sure that only the attachment points and belt guide are used as indicated in this manual

SAFETY

The Stingray Swing-out is CE marked and ECE 44.04 certified. This means that this car seat meets all relevant European safety requirements.

The Stingray Swing-out falls under the category "Special Needs Restraints" for Car Seat Groups I, 2 and 3, (9 to 36 kg). The Stingray Swing-out is approved for use in the rear seat of the car using either the Easy-fit system or Evo tension belt and possibly in the front passenger seat with the use of the EVO tension belt. These seats must be equipped with an ECE R16 approved three-point seat belt. The safest place for mounting the Stingray Swing-out is on the left-hand side of the rear seat (pavement side) (NL).

If mounting on the front passenger seat, the passenger airbag, if present, must be switched off. After the tensioning belt has been mounted and tensioned, the front seat does not have to be moved again, as otherwise the Evo tensioning belt will become loose again (NL).

No modifications can be made to the Stingray Swing-out as this will invalidate the ECE 44.04 approval.

Never leave your child unattended in the car.

Ensure that all fixings and adjustments are properly secured and check them regularly

GUARANTEE

The warranty on the Stingray Swing-out is one year after the purchase date, and this only remains valid if original parts are used for repairs. Parts that are subject to normal wear and tear are not covered by the warranty.

A 5-year warranty applies to frame breaks. Furthermore, there is a guarantee for production defects and material errors in accordance with national regulations and provisions, or as indicated in the sales and delivery conditions of the supplier where the product was purchased. These may include the provision that the warranty is subject to customers' responsibility with regard to maintenance and care of the product at the intervals specified in the manual.

The warranty can only be honored if the R82 product is used in the same country for which it was made and if the product can be identified by serial number and if the conditions of customer responsibility are met.

The warranty does not cover accidental damage, including damage caused by negligence or incorrect use. The warranty does not extend to consumable parts that are subject to normal wear and that must be replaced regularly. This warranty is void when non-original R82 parts and accessories are used or when repaired or replaced by non authorised R82 representatives.

R82 reserves the right to inspect the warranty product and the relevant documentation before agreeing to the warranty claim. The guarantee is issued by the R82 importer or by a rehabilitation dealer.

MAINTENANCE

The metal parts of the Stingray Swing-out should be cleaned with a damp cloth. The upholstery of the Stingray Swing-out seat is washable according to the washing instructions as stated on the sewn-in label.

Do not use cleaning agents that contain chloride, alcohol or aggressive ingredients.

Settings/adjustments not described in this manual may only be made by authorised R82 staff.

FITTING THE STINGRAY SWING-OUT IN A CAR

With Easyfit mounting system:

1. Before placing the Stingray Swing-out, the distance from the Easyfit mounting system (F) to the frame of the Stingray Swing-out (G) must be set to the maximum. To do this, pull out the central red button (B) and allow it to lock into its extreme position.
2. Place the protective cushions on the rear seat to protect the upholstery of the rear seat.
3. Place the Stingray Swing-out on the rear seat and slide (with a clear click) the two click systems (C) onto the Isofix anchor points. The Isofix anchoring points are located between the seat and the backrest of the rear seat. Sometimes these are well hidden.
4. Pull out the central red button (B) again and at the same time push the Stingray Swing-out towards the back of the car. Let the button click when the shortest possible length of the positioning tube has been reached. The achievable length differs depending on the seat depth of the rear seat. This positions the Stingray Swing-out.
5. Unscrew the socket head cap screws (D) of the Stingray Swing-out. The adjustable feet must be placed firmly on the floor of the car. For some cars, storage compartments are made in the bottom. Then ensure a sturdy plate between the adjustable feet and the cover plate of this storage compartment. Tighten the Allen screws after adjusting.
6. Check whether the Stingray Swing-out is positioned virtually horizontal. If this is not the case, follow steps 4, 5 and 6 again.
7. Check that the Stingray Swing-out functions properly and that it locks (clicks) in both end positions.

Removing the Stingray swing-out from the car:

1. Remove the Stingray Swing-out through the two horizontal latches (E) of the Easyfit mounting system (C). This will be the way that the Easyfit mounting system will loosen from the Isofix anchor points.
2. Carefully remove the Stingray Swing-out from the car and place it in a safe place

Fitting of the car seat must be carried out in according to this instruction.

FITTING THE STINGRAY SWING-OUT IN A CAR

With Evo tension belt

Not available.

USE OF THE STINGRAY SWING-OUT

To use the Stingray Swing-out, pull the lever (under the seat) and turn the seat 90° outwards. Pull the seat unit out until it locks into place with a clear click.

Place your child in the back of the Stingray Swing-out seat.

Optionally, your child can be secured with a 5-point chest harness (accessory).

Unlock the seat by pulling the lever again and turn the seat unit back to the direction of travel. Then push the seat inside until it falls into the lock with a clear click.

The red belt guide (3 parts) of the 3-point car belt is mounted on the seat unit as standard. Always make sure that the (car) hip belt runs tightly over the lower guide and over the hips when your child is in it. The red belt guide for the (auto) diagonal belt is standard in the lowest position (with A marked on the back of the car seat). When your child has grown it will be in the higher position (with B marked on rear car seat) fitted by releasing two bolts (C) rotate and move. Please note: the side-Impact headrest must also be set higher.


The following instructions must be followed carefully:

- Your child must be positioned correctly at the back of the seat to correctly position the 3-point belt in front of the child.
- The 5-point seat belt must be fitted correctly (see later in the user manual).

Diagonal belt:

- Place the diagonal belt over shoulder of the child.
(B) So that it passes over the shoulder of the child is walking. Optionally, an R82 protective cover (item no. 89662-1) placed over the diagonal belt at the height of the child's neck for more comfort.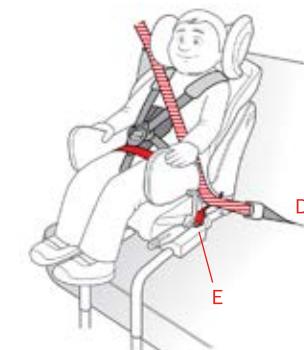

Hip belt:

The hip belt should sit well over the hips of the child and be placed through the belt guide (E)

Then place the tongue plate in the closure holder (D) to lock the 3-point belt.

ADJUST SEAT DEPTH

The seat depth can be adjusted by slightly loosening two Allen screws (D) (with Allen key 5) and moving the seat plate up to 7cm. After setting, tighten these two bolts again. The lower flap of the cushion set is held tight by a strap that runs under the seat and is secured at the rear with a folding clasp. This will have to be reset.

SEAT WIDTH

The standard seat width of seat size 1 and 2 is 28 cm and of size 3 and 4 it is 38 cm, without the pelvic supports. You can get a narrower seat width with the adjustable pelvic supports.

TILT ADJUSTMENT

The seat unit can be adjusted between 10 and 30 degrees by using the clamping lever on the side.

FOOTPLATE (accessory 95145-0 + NLPE046-1)

The footplate is fitted with a mounting rod to the front horizontal frame tube of the Stingray Swing-out size 3 and 4. The footplate can be folded up and adjusted in angle by the rotary knob on the bottom. The height can be adjusted by loosening the Allen screw and adjusting it to the correct height. If the foot plate is set too high, the turning part will touch the foot plate. The footplate can therefore not be used in this position.


ADJUSTABLE PELVIC SUPPORT (accessory 87402XX)

The standard hip supports have three possible width options.

They can be adjusted by the addition and removal of the bolt and spacers.

The adjustable hip supports use flip up levers to give greater adjustment possibilities.

Always ensure that the car seat belt runs over the users hips and not over the hip supports.

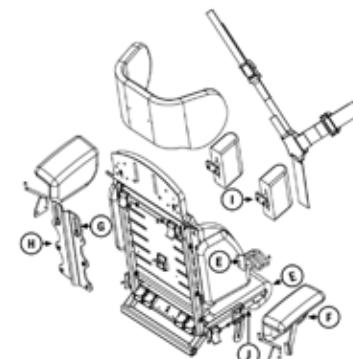SIDE IMPACT HEADREST ~ (standard)

The Side Impact headrest (SI) can be adjusted in total by approximately 10 cm in height.

Loosen two bolts (G) (with Allen key 6) and slide the SI headrest. Optionally, the mounting plate can still be moved in height by loosening the 4 bolts (H) and moving the mounting plate one step higher. This causes the SI headrest to rise above the seat.

ANATOMICAL NECK CUSHION (8720831)

As an additional support, the Side-Impact headrest can be fitted with an anatomical neck cushion with folding clasps that provide additional support for the child's head and neck

SIDE SUPPORTS (NLPE040/041)

The side supports (I) are placed in the slots in the backrest. Then fix the side supports using the clamp levers on the backrest.

OUTER KNEE SUPPORTS (NLPE043/044)

The outer knee supports are fitted on the outside of the seat plate by means of two bolts (J). These knee supports are provided with a cover.

REAR SEAT PROTECTION (NLPE011)

I.C.M. PROTECTION PILLOWS (NLSW071)

The rear seat protector and protective pads are used to protect the rear seat. Place the protective cushions on the rear seat (backrest and seat) and the rear seat protector on the outside of the protective cushion. The end of the plate is level with the side of the rear seat. For example, the back seat is protected when the car seat is extended.

5-point chest harness (accessory NLPE047 (Small) en NLPE048 (Large))

The 5-point chest harness is especially suitable for the Stingray Swing-out and is the only additional fitting accessory that is permitted with this car seat. This 5-point chest harness is not a safety belt but is intended to further secure your child in the seat unit. The 5-point belt is fitted on the bottom and rear of the seat unit with folding buckles.

1. Open the 5-point harness by clicking on A and B from the harness attachment. (Fig. 1).


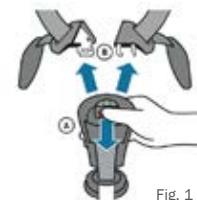
Fig. 1

2.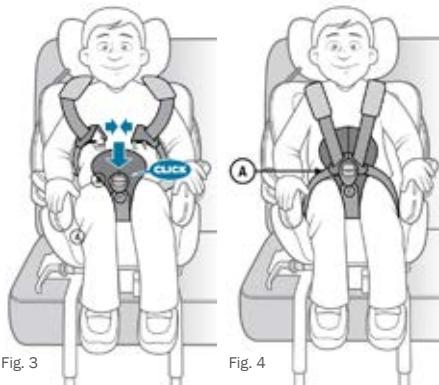

Please check that the shoulder straps are not twisted (Fig. 5)

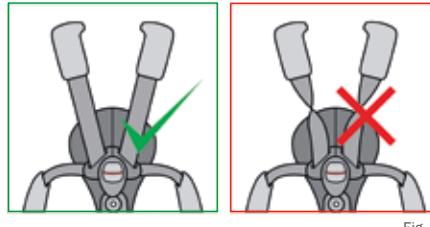
Fig. 5

4. Then pull the straps A and B (Fig. 6)

Tighten to secure the 5-point belt.
Make sure there is a maximum of 1cm of clearance between your child and the shoulder straps (Fig 7).

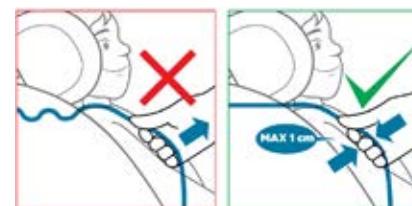
Fig. 7

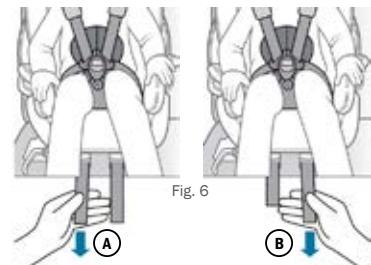
Fig. 6

Fig. 8

5. Then place the 3-point car seat belt (B) over your child and close it. Make sure that the two seat belts are placed correctly over the child (Fig 8). Ensure that the seat belt is not positioned too high when mounted on the front seat. See fig 9 (NL).

Fig. 8

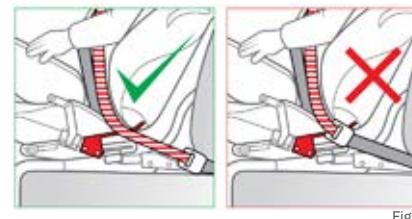
Fig. 9

The 3-point car seat belt must always be used. Even without a child in it.


Other fixing systems are not to be used in the Stingray swing-out as it will invalidate its ECE-44.04 approval.


PRODUCT IDENTIFICATION LABELS

You will find these labels on the Easyfit frame

- A) Manufacturer label with batch number
- B) Manufacturer with serial number (SN)
- C) Approval number
- D) Distributor contact details

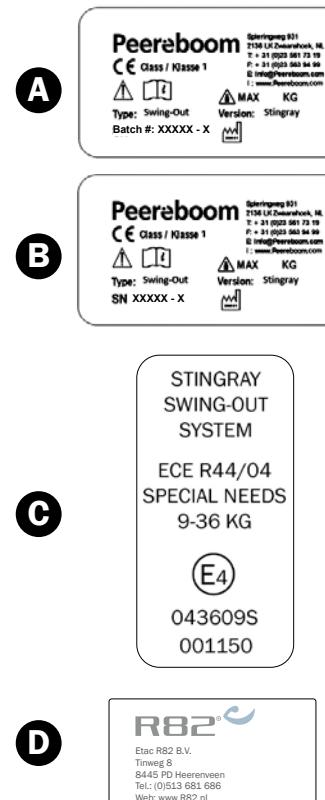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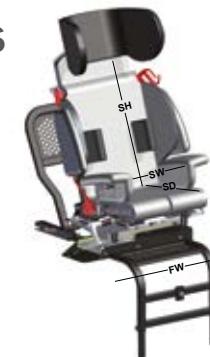DECLARATION OF CONFORMITY: MANUFACTURER

Company name	: Peereboom BV
Address	: Spieringweg 931
Postal code	: 2136 LK
Place	: Zwaanshoek
Country	: The Netherlands
Phone	: 0031 (0)23 561 73 19
Fax	: 0031 (0)23 563 94 99
E-mail	: Info@Peereboom.com
Website	: www.Peereboom.com

DESCRIPTION AND IDENTIFICATION OF THE PRODUCT

Name	: Swing-Out
Brand	: Stingray Swing-out size 1/2/3/4
Function	: Car seat for children with a body weight of 9 to 36 kg
Serial number	: xxxx

COMPLIANCE

The manufacturer declares that the above mentioned product fulfills all relevant provisions of
Medical Devices Regulation 2017 / 745
for the design and manufacture

NEN-EN ISO 9001:2015; NEN-EN ISO 14001:2015;
EN 12182:2012; EN-ISO 14971:2012; ECE R44/04;
Flammability acc. to EN 71-2; Toxicity acc. to EN 71-3

Place: Zwaanshoek
The Netherlands
Dated: 8 April 2020

Name: H.A. Peereboom
Function: Company Director
Signature:

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Etac R82 GmbH
Etac R82 Srl