WINTHER-REHA

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GB INTRODUCTION

Congratulation on purchasing the Winther-Reha bicycle-trailer.

This quality product is made especially to make it possible to bring children with a handicap on the bicycleride.

The plastic hull is provided with an adapter for mounting a R82 Panda Futura Seat size 1,2,2½ or 3. The adapter can be positioned as needed, to get the best comfort for the child.

The Winther-Reha bicycle trailer is produced according to new standards for security, comfort and versatility.

- The plastic base is created with a low seating position and a wide wheel base to deflect the impact and provide maximum safety for its passengers.
- A comfortable ride is secured by the angling possibilities, the suspension wheels and ample leg room.
- The trailer has a large luggage room and a total carrying capacity of 50 Kg.
- The front wheel and the push handle are supplied as standard, allowing the Winther-Reha bicycle trailer to be used as a push buggy.
- The use of aluminium and plastic brings the weight of the Winther-Reha bicycle trailer without the Panda Futura seat down to 15 kg.

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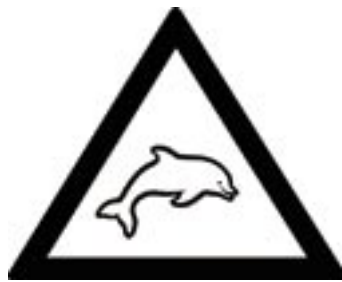
SECURITY CHECK Read before use

- Make sure the trailer is correctly assembled and that the components are tightly and accurately joined.
- Never exceed a load of 45 kg in the trailer incl. the Panda Futura seat and the child.
- Make sure that the Panda Futura seat is correctly mounted in the adapter. The lock, the green knob, must lock correctly in the adapter (it must click in the adapter).
- Check the back angling of the Panda Futura seat. Tighten it firmly.
- The child must wear a safety belt in the Panda Futura seat, a 5 point fixation is recommendable.
- Fasten the safety strap round the bicycle frame. Never use the bicycle without the safety strap and the safety cotters properly fitted.
- The child should always wear an approved bicycle helmet.

The weight distribution in the trailer is important.

If heavy luggage is placed in the luggage compartment behind the Panda Futura Seat, this may displace weight to the rear and it may be necessary to move some of the luggage to the front of the trailer, to prevent the trailer from tipping.

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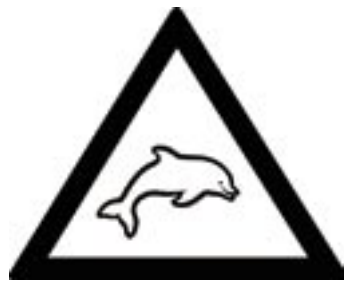
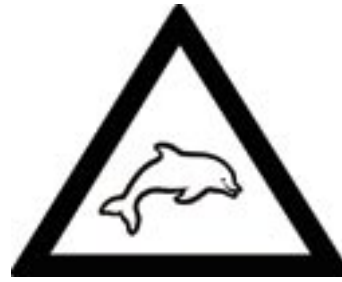
GB RIDING TIPS

- The front and the insect net always has to be fitted when riding. The net will protect against small stones, insects ect.
- Adapt your speed to road conditions and never exceed 20 mph/30 kph. Take care when taking sharp bends and corners.
- The bicycle's braking distance will be larger when towing a trailer.
- Remember the width of the trailer.
- Make sure the trailer brake is on and that the bicycle is properly parked, when the child is getting in and out of the trailer.
- Never leave the child alone behind in the trailer.
- Take the child out of the trailer, during a brake in bright sunshine.

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GB MAINTENANCE

The Winther-Reha bicycle trialer is constructed of maintenance-free materials.

It's life can however be extended by following the advice below.

- Avoid standing the trialer in direct sunlight for longer periods.
- Prevent dirt from getting into important connections (whells, tow bar, coupling etc.)
- Wash the top and the plastic hull with soap and water and a soft brush or sponge while they are mounted on the trailer.
- Machine wash at max. 30°. Important! The top has to be mounted immediately after washing to avoid crumpling of the windows.
- The cushion covers of the Panda Futura seat are detachable and can be washed at 30°.
- The tyre pressure should be 35-45 PSI.
- Fix loose Velcro with ordinary adhesive, the surface can be washed with spirits.

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GUARANTEE

The Winther-Reha bicycletrailer is guaranteed on the fol-lowing terms:

- Frame and components are guaranteed for one year from date of puschase, except for wear to tyres and inner tubes.
- The guarantee is restricted to the repair or in certain cases replacement of components and excludes transport and labour, compensastion for consequential damage will not be paid.
- The guarantee is restricted to the original purchaser. All claims must be accompanied by proof of purchase.
- The Winther-Reha bicycletrailer is only to be used as a bicycle trialer, push-buggy or jogging carriage. Do not use the trailer with mopeds, motorcycles and other vehicles.
- To attach a bicycle to the Winther-Reha bicycletrailer use only its original Winther-coupling, a WEBER coupling or another officially approved coupling.
- Any dispute shall be settled in a danish court and cording to danish law.

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GB JOINING THE SAFETY BARS

Join the rollover bar and the front bar with the two screw knobs.

Make sure the sides of the top are fitted to the front bar before joining.

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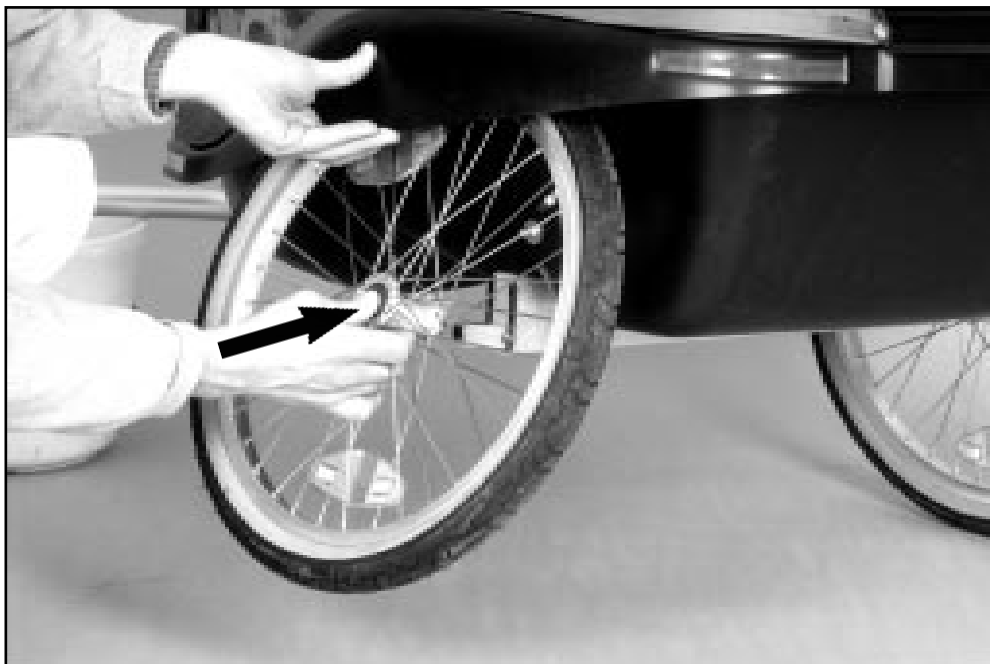
GB FIXING THE FRONT BAR

Fix the front bar to the plastic hull with the strong nylon strap.

Tighten off the nylon strap with the velcro fastener.

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QR-akslen er forsynet med en snaplås. Knappen i navet trykkes ind og hjulet skubbes ind over akslen.

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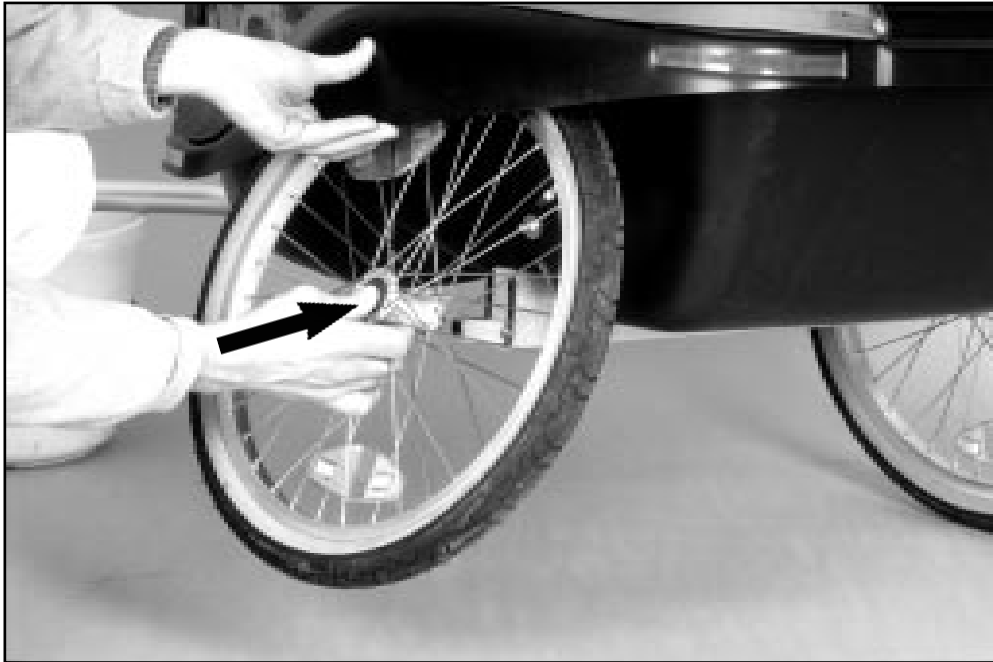
GB MOUNTING THE WHEELS

The wheel hub has a spring lock. Press in the button on the wheel hub and fit the hub axle into the axle aperture.

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GB DEMOUNTING THE WHEELS

Press in the button on the wheel hub and pull the wheel carefully against the plastic base until it clicks. Push the wheel back against the axle. Press in the button on the wheel hub and pull out the hub axle. Remove the wheel.

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GB MOUNTING THE TOW-BAR / FRONT WHEEL

Move the towbar or front wheel into the T-frame. Fit the safety cotter and the lock with the spring bar.

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GB MOUNTING THE TOW- BAR / FRONT WHEEL(2)

Turn the lever upwards until it clicks. The lock pawl is then engaged.

Note! Adjust the plug axle/nut to span the other tube firmly around the towbar.

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Monter koblingen permanent på cyklens venstre bagstang, så langt bagud som muligt.

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GB MOUNTING THE BICYCLE COUPLING

The coupling should be permanently mounted on the left-hand rear wheel stay, as far back as possible .

The large hook is fitted on the lower rear wheel stay. The small is to be fitted on the upper rear wheel stay. There is four differen positions depending on the bicycle frame type.

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GB MOUNTING THE
BICYCLE COUPLING

Use a 4 and a 5 mm Allen key to fix the Allen screws.

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GB MOUNTING THE TOWBAR

Push the towbar over the coupling flexi-joint. Fit the safety cover from above through the hole in the towbar and lock with the spring bar. Pull the safety strap round the frame in front of the coupling and lock off with the snap hook in the ring under the towbar.

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GB MOUNTING THE TOWBAR

Never use the trailer without fitting the safety strap correctly.

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GB MOUNTING THE TOP SIDES

Pull the top sides over the front bar and fix them to the plastic shell with the press-studs and the velcro.

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GB MOUNTING THE TOP/
BACK FLAP

Secure the hooks of the back flap to the rear end of the plastic shell and pull the top forwards over the front bars. Secure the top front to the front of the plastic shell.

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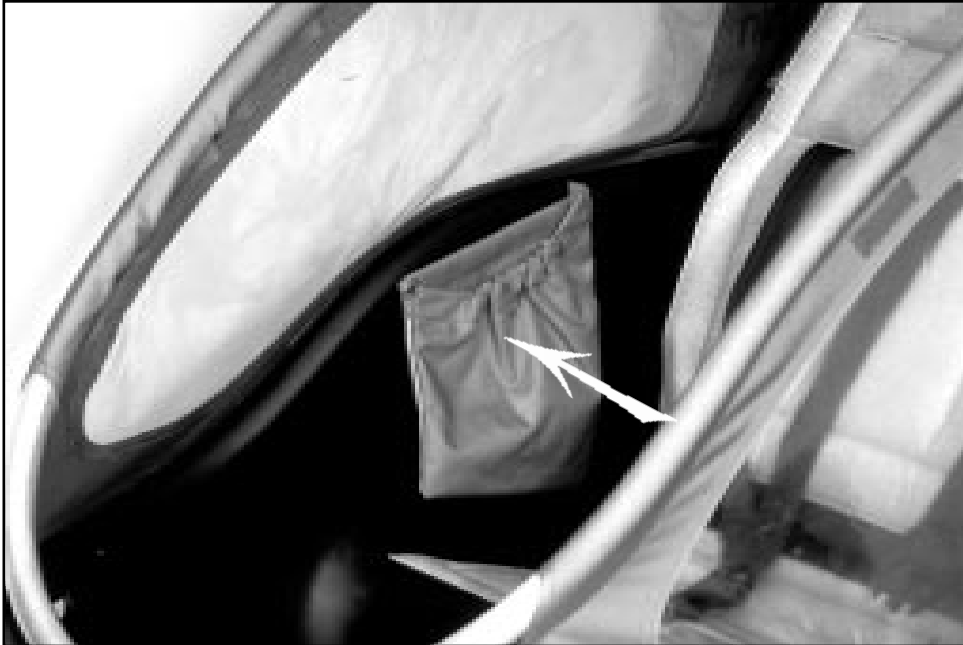
GB MOUNTING THE TOP

Make sure that the top is properly positioned and tautened over the front bars. Tauten the top by pulling it down at the sides and fixing it with the velcro connectors to the sides. Never ride with the front rolled up.

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GB MOUNTING THE POCKETS

Mount the pockets to the inner side of the plastic sheel using the press studs.

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GB MOUNTING THE FLAG

Mount the flag pole to the top of the roll-over bar through the ring in the top.

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GB MOUNTING THE PUSH HANDLE

Mount the push handle to the roll-over bar with the two screw knobs.

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GB TIPPING THE TOP

Unhook the hooks at the front of the top. Open the velcro connectors at the front. Now open the press-studs along the trailer sides.

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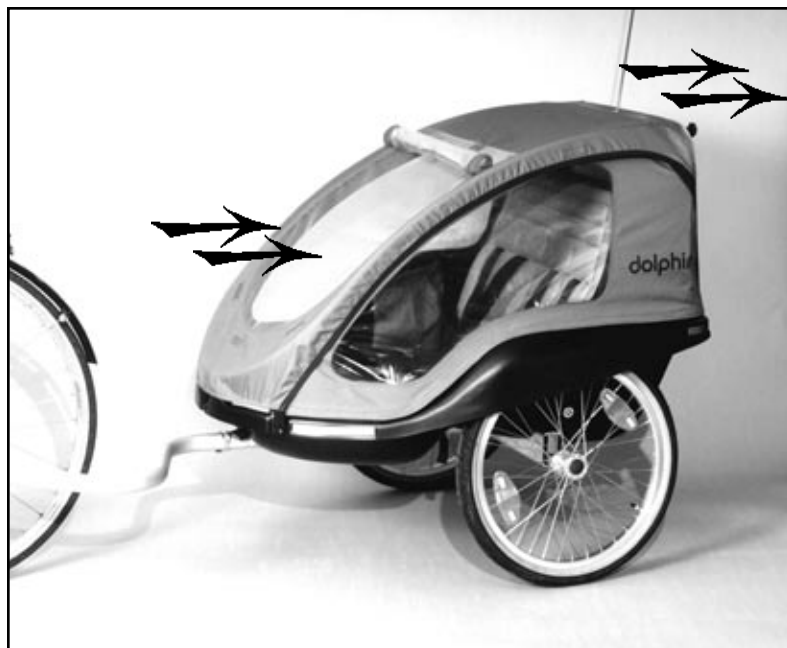
GB VENTILATION OF THE COMPARTMENT

The trialer is provided with ventilation holes at the front and rear of the passenger compartment, giving permanent ventilation when the front window is closed.

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GB INSECTNET

Open the front window by the hooks, roll it up and fasten it with the press-studs.

The ventilation through the net will be good and even better if the rear flap is opened too.

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GB OPENING THE TOP

Open the front hooks, roll it up and secure it with the rub-ber band on the top.

The frontside and the insectnet always have to be closed during a ride. They protect agianst small stones, insects and similar.

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GB OPENING THE BACK

Open the rear hooks and the back of the top is open.

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GB THE BRAKE

Activate the brake by pressing the handle down to STOP.

The brake should always be activated, when the child is getting in and out of the trailer.

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GB FOLDING THE TRAILER

Remove the top.

Remove the wheels and place them inside the trailer. (See page 19)

Remove the push bar and place it in the trailer.

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GB FOLDING THE TRAILER

Unscrew the finger screws joining the front and rollover bars and fold down the rollover bar.

Rescrew the finger screws into the frontbar.

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Trailer: Strengthened polyethylen plastic
Frame and towbar: Hardened aluminium tube
Coupling: Casted aluminium and polyurethan
flexible link
Top: Nylon
Wheel: 20x1,75 alu rim with stainless spokes
Hub: Aluminium with closed ball bearings

Width	79 cm
Height	115 cm
Height of the push bar	104 cm
Wheel base:	77 cm
Length with towbar:	162 cm
Length with front wheel:	139 cm
Weight	15 ½ kg
Max load:	45 kg

(Approved according to national standards)

MANUFACTURER:

A. Winther A/S
DK-8653 THEM
FAX. (+45) 86848528
E-mail: win @ a-winther.dk

Website: [www . winther-bikes.com](http://www.winther-bikes.com)

DEALER:

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R82 GmbH