

wombat[™]


Exploded views / Part lists

Rev. date (09.2014)



Contents



Wombat seat unit size 1+2	3
Wombat seat unit size 3.....	6
Wombat Basic with gas spring size 1+2	9
Wombat Upgrade Frame size 1+2	11
Wombat Upgrade Frame size 3.....	13
Wombat Upgrade with power unit size 1+2+3	15
Central braking system size 1+2+3.....	17
Hip support and Knee support.....	19
Hip support, Lower back support, Fix locks, Armrest, Adjustable Handbrace and Anti tip safety for size 3	21

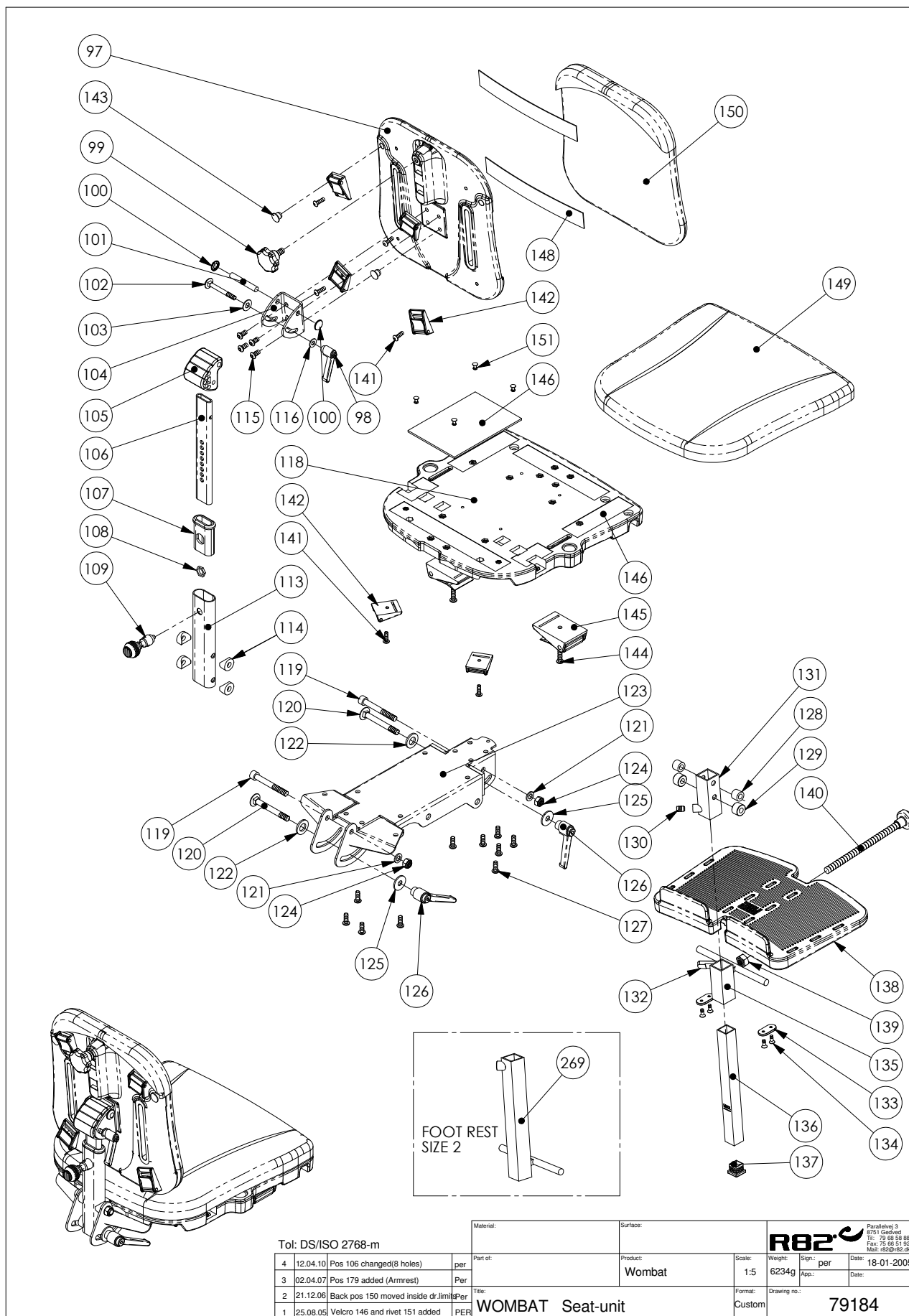

Position	No.	Description	Quantity	Unit of Measure	Ean code
97 *	82943	BACK SIZE 1, WOMBAT	1	PCS	5707292125989
	82945	BACK SIZE 2, WOMBAT	1	PCS	5707292126009
98	3502200	TIGHTENING HANDLE 42MM M6 BUSH	1	PCS	5707292146861
99	3501000	STAR GRAP Ø40 M8X15MM BLACK	1	PCS	5707292146700
100	1109510	STARLOCK WASHER Ø8	2	PCS	5707292140456
101	80920	STICK '8*54	1	PCS	5707292111364
102	1905545	BOLT M6*60, FZB	1	PCS	5707292142306
103	1105080	STAINLESS WASHER 8MM	1	PCS	5707292140364
104	82915	U-FITTING FOR BACK	1	PCS	5707292125729
105	82947-XX	BACK TILTING DEVICE	1	PCS	
106 *	82916	BACK COLUMN SIZE 1	1	PCS	5707292125736
	82928	BACK COLUMN SIZE 2	1	PCS	5707292125835
107	82948	BUSH 10/30, WOMBAT	1	PCS	5707292126023
108	1785000	NUT 12MM HEXAGON, FZB	1	PCS	5707292141583
109	9500001	LOCK WOMBAT	1	PCS	5707292404527
113	82917	BACK COLUMN	1	PCS	5707292125743
114	82929	DISTANCE PIECE	4	PCS	5707292125842
115	0641200	BUTTON HEAD SCREW 6*12	4	PCS	5707292139252
116	1125618	WASHER 6MM	1	PCS	5707292140371
118 *	82941	SEAT SIZE 1, WOMBAT	1	PCS	5707292125965
	82942	SEAT SIZE 2, WOMBAT	1	PCS	5707292125972
119	0603665	INSEX SCREW 8*65	2	PCS	5707292138286
120	1905710	BOLT M8*70 FZB WO/NUT	2	PCS	5707292142368
121	1103200	FAZET WASHER 8MM	2	PCS	5707292140296
122	1103500	FAZET WASHER 12MM	1	PCS	5707292140326
123	82898	BASIS PLATE	1	PCS	5707292390691
124	1607001	SELF-LOCKING NUT 8MM	2	PCS	5707292401281
125	1108000	WASHER 8MM	1	PCS	5707292140388
126	3502300	TIGHTENING HANDLE 65MM M8 BUSH	2	PCS	5707292146885
127	0641600	BUTTON HEAD SCREW 6*16	10	PCS	5707292139269
128	1350022	DISTANCE PIECE 12*15*8,2MM	2	PCS	5707292140906
129	82934	DISTANCE PIECE	2	PCS	5707292125897
130	0701000	STOP SCREW WITH CRATER 8*10	1	PCS	5707292139870
131	82926	TURNING FITTING	1	PCS	5707292125811
132	3502516	TIGHTENING HANDLE 45MM M8*16	1	PCS	5707292146915
133	82950	HOLLOW PLATE 16*32, WOMBAT	2	PCS	5707292126030
134	1940017	SHEET METAL SCREW UHX4,2*13FZB	4	PCS	5707292142597
135/269 *	82925	SUPPORT FITTING, SIZE 1	1	PCS	5707292125804
	82737	SUPPORT FITTING, SIZE 2	1	PCS	5707292399366
136	82927	PIPE FOR FOOTPLATE, WOMBAT	1	PCS	5707292125828
137	2504500	FERRULE E20*1,0-2,0 INT.BLACK	1	PCS	5707292144126
138 *	82944	FOOTPLATE SIZE 1, WOMBAT	1	PCS	5707292125996
	82940	FOOTPLATE SIZE 2, WOMBAT	1	PCS	5707292125958
139	1607500	LOCK NUT 10 MM	1	PCS	5707292141385
140	3502780	HANDLE 28MM,M10*160MM THREAD	1	PCS	5707292363145
141	0658012	BUTTON HEAD M5*12 FZB	2	PCS	5707292139382
141	0658012	BUTTON HEAD M5*12 FZB	4	PCS	5707292139382
142	0445025	25MM PRESS BUCKLE, BLACK	6	PCS	5707292465160
143	2501600	FERRULE D15*1,0 BLACK	2	PCS	5707292143969

144	0641620	BUTTON HEAD SCREW 6*20	2	PCS	5707292139276
145	0442070	FIXLOCK CLAMP 40 MM W/HOLE	2	PCS	5707292136398
146	4913034	APLIX 30 MM, HOOK BLACK W/TAPE	0,74	METER	5707292151858
148	4913034	APLIX 30 MM, HOOK BLACK W/TAPE	0,53	METER	5707292151858
149 *	95021-2XX	SEAT CUSHION SIZE 1	1	PCS	
	95022-2XX	SEAT CUSHION SIZE 2	1	PCS	
150 *	95031-2XX	BACK CUSHION, SIZE 1	1	PCS	
	95032-2XX	BACK CUSHION, SIZE 2	1	PCS	

	XX	CHOOSE COLOUR CODE: 20: RED, 50: BLUE
*		CHOOSE SIZE

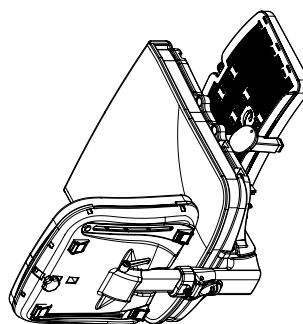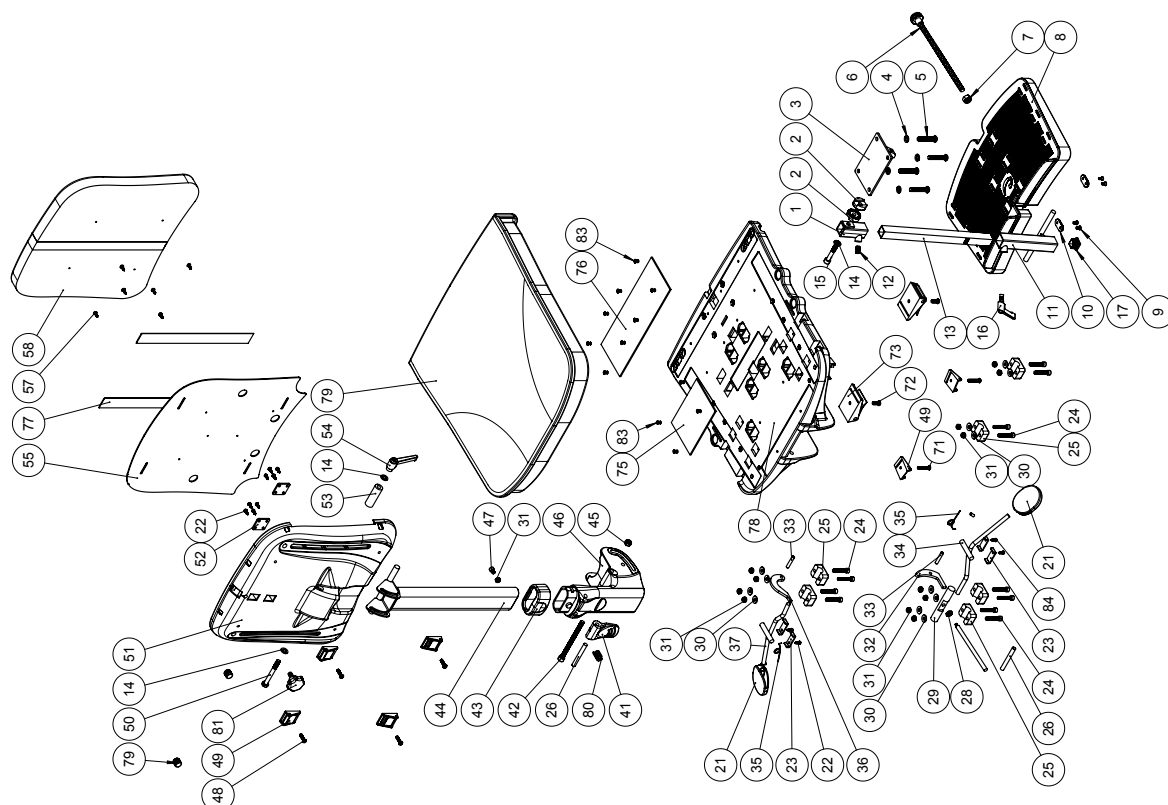

R82 nothing compares to a smile	Model 1 16 20 21 85 Swift JYBA DKKK	Created by [ja]	Revision by [ja]	Revision date 14-08-2014
		Project Wombat 3 Seat	Item no. 79198	

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	82735	FIX FOR FOOTREST	1	PCS	5707292399335
2	74061	TOOTH WASHER Ø30	2	PCS	5707292373816
3	82734	PLATE COMPLETE	1	PCS	5707292398932
4	1101000	CENTER SPRING WASHER 8MM	4	PCS	5707292140173
5	0661145	BUTTON HEAD 8*45 FZB	4	PCS	5707292139535
6	3502795	KNOB 28MM,M10*??MM THREAD	1	PCS	5707292402110
7	1607500	LOCK NUT 10 MM	1	PCS	5707292141385
8	82752	FOOTPLATE SIZE 3, WOMBAT	1	PCS	5707292401373
9	1940017	SHEET METAL SCREW UHX4,2*13FZB	4	PCS	5707292142597
10	82950	HOLLOW PLATE 16*32, WOMBAT	2	PCS	5707292126030
11	82737	PIPE FITTING LONG, WOMBAT 3	1	PCS	5707292399366
12	0701000	STOP SCREW WITH CRATER 8*10	1	PCS	5707292139870
13	82787	PROFILE	1	PCS	5707292401472
14	1103200	FAZET WASHER 8MM	1	PCS	5707292140296
14	1103200	FAZET WASHER 8MM	2	PCS	5707292140296
15	0603660	INSEX SCREW 8*60	1	PCS	5707292138279
16	3502516	TIGHTENING HANDLE 45MM M8*16	1	PCS	5707292146915
21	3470336	RELEASE BUTTON 5*12MM,ROUND	2	PCS	5707292397607
22	1940032	ECO-SIGHT Z M3,5X12 RUSTPROOF	4	PCS	5707292397584
22	1940032	ECO-SIGHT Z M3,5X12 RUSTPROOF	8	PCS	5707292397584
23	112307	LOCKPLATE	4	PCS	5707292591722
24	0602435	INSEX SCREW 6*35	12	PCS	5707292138132
25	2521840	PIPE HOLDER F/16MMSQUARE,BLACK	6	PCS	5707292399236
26	0530005	CYL.PIN Ø8*66 MM	2	PCS	5707292397805
27	2203322	BLOC-O-LIFT 684074 200N SOLID	1	PCS	5707292398147
28	1760800	COMPENSATING NUT 8*1,0 FINEP.T	1	PCS	5707292141521
29	82741	SHAFT	1	PCS	5707292400109
30	1106000	WASHER 6MM	12	PCS	5707292140371
31	1606500	LOCK NUT 6 MM	12	PCS	5707292141361
31	1606500	LOCK NUT 6 MM	1	PCS	5707292141361
32	82746	RELEASE DISK	1	PCS	5707292398895
33	0530003	CYL.PIN Ø6*32 MM	2	PCS	5707292397638
34	82772	GRIP FOR BACKTIP	1	PCS	5707292398857
35	0310519	TWIST SPRING	2	PCS	5707292400321
36	82728	RELEASE DISK	1	PCS	5707292398864
37	82769	GRIP FOR SEATTIP	1	PCS	5707292398871
41	82795	LOCKGRIP	1	PCS	5707292400079
42	0603975	INSEX SCREW 8*110	1	PCS	5707292138330
43	82796	LOCKCOVER	1	PCS	5707292400055
44	82765	BACKPILE	1	PCS	5707292398918
45	1607000	LOCK NUT 8MM, LARGE PACK	1	PCS	5707292141378
46	82797	MOUNTING	1	PCS	5707292400048
47	0602188	INSEX M6*8, FZB	1	PCS	5707292138071
48	0640920	BUTTON HEAD M5*20 FZB	4	PCS	5707292381033
49	0442065	FIXLOCK BUCKLE 25MM BLACK	2	PCS	5707292136381
49	0442065	FIXLOCK BUCKLE 25MM BLACK	4	PCS	5707292136381
50	1907100	STEEL BOLT 8*70MM	1	PCS	5707292142467
51	82751	BACK MOUNTED WITH SCREWS	1	PCS	5707292400123
52	82781	MOUNTING PLATE	2	PCS	5707292398840


53	82747	SHAFT	1	PCS	5707292400130
54	3502300	TIGHTENING HANDLE 65MM M8 BUSH	1	PCS	5707292146885
55	82794	BACKPLATE	1	PCS	5707292400116
57	1940010	PLASTITE 60, 2,9*6,5	6	PCS	5707292142573
58	95033-218	BACK CUSHION SIZE 3,BLACK	1	PCS	5707292398703
71	0640930	BUTTON HEAD M5*30 FZB	2	PCS	5707292139221
72	0641600	BUTTON HEAD SCREW 6*16	2	PCS	5707292139269
73	0442070	FIXLOCK CLAMP 40 MM W/HOLE	2	PCS	5707292136398
75	4935400	100 MM APLIX,BLACK,HOOK W/TAPE	0,145	METER	5707292409621
76	4935400	100 MM APLIX,BLACK,HOOK W/TAPE	0,27	METER	5707292409621
77	4913034	APLIX 30 MM, HOOK BLACK W/TAPE	0,62	METER	5707292151858
78	82739	SEAT MOUNTED WITH SCREWS	1	PCS	5707292400024
79	95023-218	SEAT CUSHION SIZE 3, BLACK	1	PCS	5707292398697
80	0310163	COMPRESSION SPRING1,3*12RUSTPR	1	PCS	5707292402790
81	3501000	STAR GRAP Ø40 M8X15MM BLACK	1	PCS	5707292146700
82	2501600	FERRULE D15*1,0 BLACK	1	PCS	5707292143969
75+76+77	4913034	APLIX 30 MM, HOOK BLACK W/TAPE	1	METER	5707292151858

WOMBAT BASIC
MANUAL OPERATION
95-122-XX

Pos. 27 and 39
not included

2201700

1108000

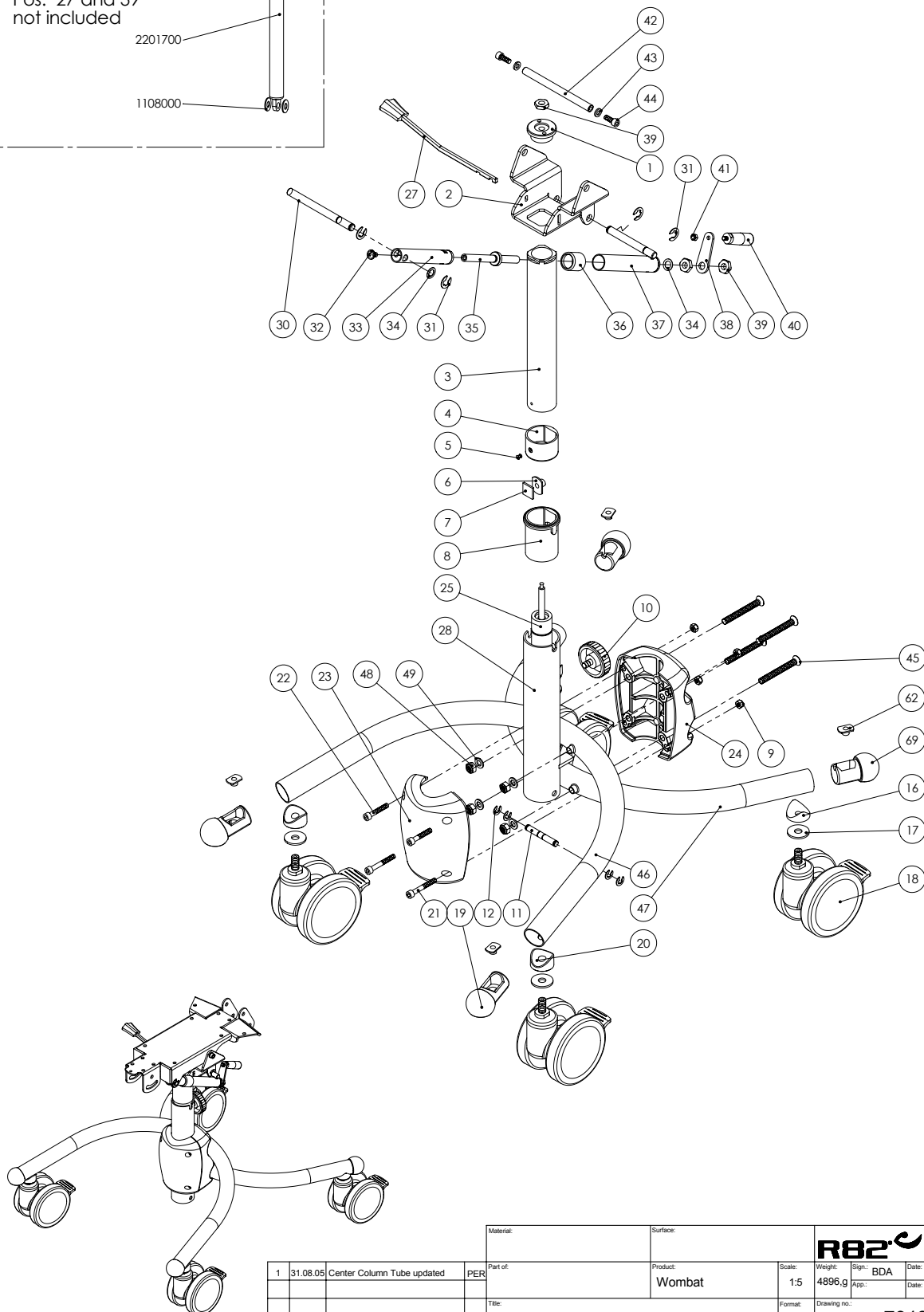


1	31.08.05	Center Column Tube updated	PER

Material:	Surface:	R82[®] Parallelvej 3 8751 Gedved Tlf.: 75 68 58 88 Fax: 75 68 51 82 Mail: r82@r82.dk			
Part of:	Product:	Scale:	Weight:	Sign:	Date:
	Wombat	1:5	4896g	BDA	16-05-2001
Title:	Format:	Drawing no.:			
WOMBAT BASIC - with gas spring	Custom	79181			

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	82996	CHAPTER M30*1	1	PCS	5707292126382
2	82987	TILT FITTING, WOMBAT BASIC	1	PCS	5707292126290
3	82984	INNER COLUMN	1	PCS	5707292126269
4	82957	BOTTOM BUSH	1	PCS	5707292126061
5	0504600	POP RIVET 3,2*6,0	1	PCS	5707292137272
6	1600520	STAINLESS NUT M8 W/FLANGE	1	PCS	5707292141132
7	82997	EXTENSIONPLATE	1	PCS	5707292126399
8	82956	GUIDE BUSH, WOMBAT	1	PCS	5707292126054
9	1603000	STEEL NUT 6 MM	4	PCS	5707292141279
10	3470340	HANDLE Ø52 WITH M8*20 SCREW	1	PCS	5707292146649
11	80923	ø8*50 LOCKING PIN, WOMBAT	1	PCS	5707292111395
12	1145070	LOCK WASHER 7MM	4	PCS	5707292140586
14	1600525	STAINL. WELDING NUT W/FLANGE	4	PCS	5707292141149
16	82894	DISTANCE PIECE 30*3,4*10,4	2	PCS	5707292140951
17	1108050	WASHER M10, DIN9021	4	PCS	5707292140401
18	3006930	100 MM GREY DOUBLE WHEEL	4	PCS	5707292145918
19	82959-30	BALL Ø40, GREEN	2	PCS	5707292232274
20	82958-30	DISTANCE PIECE	2	PCS	5707292232236
21	0602445	INSEX M6*45 FZB	2	PCS	5707292138156
22	0602440	INSEX M6*40 FZB	2	PCS	5707292138149
23	82955-XX	FRONT BODY	1	PCS	
24	82954-XX	BACK-BODY, WOMBAT	1	PCS	
25	2200505	BLOC-O-LIFT 729655 250N FJED.	1	PCS	5707292143136
27	2200017	ARM FOR HEIGHTADJUSTMENT	1	PCS	5707292143020
28	82719	CENTRE COLUMN, WOMBAT BASIC	1	PCS	5707292126252
29	1310712	NYLON WASHER, NATURE	1	PCS	5707292356345
30	82995	STOP PIN, Ø10 MM	1	PCS	5707292126375
31	1145090	LOCKING DISK 9MM,DIN6799,RUSTP	4	PCS	5707292140609
32	0641100	B.H. W/FLANGE M6*10	1	PCS	5707292139245
33	82986	INNER PIPE, WOMBAT BASIC	1	PCS	5707292126283
34	1310640	WASHER, NATURE Ø16*10*0,5MM	2	PCS	5707292140760
35	82993	THREAD PIPE M10*1	1	PCS	5707292126351
36	72215	FERRULE FOR PANTHER	1	PCS	5707292106667
37	82985	OUTER PIPE, WOMBAT BASIC	1	PCS	5707292126276
38	82988	PLATE FOR HANDLE, WOMBAT BASIC	1	PCS	5707292126306
39	1761000	COUNTER NUT 10*1,0 FINE THREAD	3	PCS	5707292141538
40	3520000	HANDLE, TURNABLE M5*7, BLACK	1	PCS	5707292147141
41	1606300	LOCK NUT 5 MM	1	PCS	5707292141354
42	82606	AXLE FOR TILT FITTING	1	PCS	5707292364197
43	1103000	FAZET WASHER 6MM	2	PCS	5707292140289
44	0602200	INSEX SCREW 6*16	2	PCS	5707292138095
45	0625071	INSEX UNDERS. M8*70, FZB	4	PCS	5707292139108
46	82981	FRONT PIPE, WOMBAT BASIC	1	PCS	5707292126238
47	82982	BACK PIPE, WOMBAT BASIC	1	PCS	5707292126245
48	1607000	LOCK NUT 8MM, LARGE PACK	4	PCS	5707292141378
49	1108000	WASHER 8MM	4	PCS	5707292140388
69	82829-40	BALL Ø40, YELLOW	2	PCS	5707292231864

XX	CHOOSE COLOUR CODE: 20: RED, 50: BLUE
----	---------------------------------------

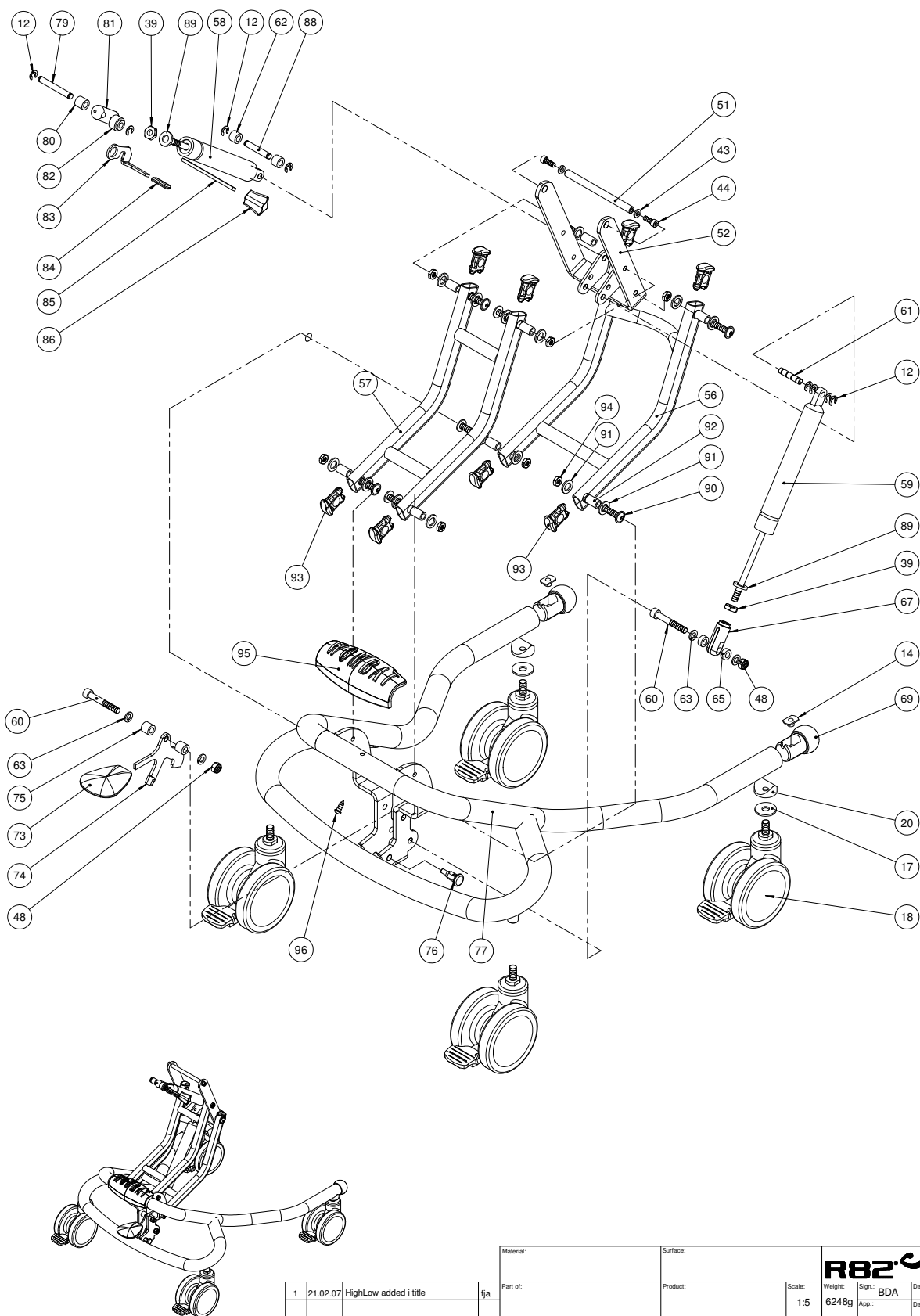


1	21.02.07	HighLow added i title	fja

Material:	Surface:	R82[®] Parallelvej 3 8751 Gedved Tlf: 79 68 58 88 Fax: 79 68 51 92 Mail: r82@r82.dk			
Part of:	Product:	Scale:	Weight:	Sign:	Date:
		1:5	6248g	BDA	17-10-2004
Title:	Format:	Drawing no.:			
Wombat Upgrade Frame / HighLow			Custom	79183	


Position	No.	Description	Quantity	Unit of Measure	Ean code
12	1145070	LOCK WASHER 7MM	8	PCS	5707292140586
14	1600525	STAINL. WELDING NUT W/FLANGE	2	PCS	5707292141149
17	1108050	WASHER M10, DIN9021	2	PCS	5707292140401
18	3006930	100 MM GREY DOUBLE WHEEL	4	PCS	5707292145918
20	82894	DISTANCE PIECE 30*3,4*10,4	2	PCS	5707292140951
39	1761000	COUNTER NUT 10*1,0 FINE THREAD	2	PCS	5707292141538
43	1103000	FAZET WASHER 6MM	2	PCS	5707292140289
44	0602200	INSEX SCREW 6*16	2	PCS	5707292138095
48	1607000	LOCK NUT 8MM, LARGE PACK	2	PCS	5707292141378
51	82806	AXLE	1	PCS	5707292125200
52	82924	TILT BRACKET, WOMBAT	1	PCS	5707292125798
56	82906	SINGLE SUPPORT BAR, WOMBAT	2	PCS	5707292125637
57	82921	SUPPORT BAR, WOMBAT	1	PCS	5707292125781
58	2203506	BLOC-O-LIFT GASSPRING 2745HU	1	PCS	5707292143297
59	2205300	BLOC-O-LIFT GASSPRING 5251HS	1	PCS	5707292143389
60	0603600	INSEX SCREW 8*55	2	PCS	5707292138262
61	80924	Ø8*40 SPEC LOCKING PIN, WOMBAT	1	PCS	5707292111401
62	1350021	DISTANCE PIECE 11*15*8,2MM	2	PCS	5707292140890
63	1103200	FAZET WASHER 8MM	4	PCS	5707292140296
65	1350000	DISTANCEPART BLACK 6*15*8,2 MM	2	PCS	5707292140876
67	6118822	HORIZONTAL RELEASE	1	PCS	5707292186744
69	82829-XX	BALL Ø40	2	PCS	
73	3470335	RELEASE BUTTON Ø8, 5*8MM	1	PCS	5707292146632
74	82914	PEDAL, WOMBAT	1	PCS	5707292125712
75	1350023	DISTANCE PIECE, BLACK	2	PCS	5707292140913
76	0380183	SPEC. HOUSE FOR LOCK, COMPLETE	1	PCS	5707292136046
77	82920	FRAME FOR WOMBAT	1	PCS	5707292125774
79	80922	LOCKING PIN, WOMBAT	1	PCS	5707292111388
80	1350024	DISTANCE PIECE 14*15*8,2MM	1	PCS	5707292140920
81	6118821	HORIZONTAL RELEASE, WOMBAT	1	PCS	5707292186737
82	82930	BUSH FOR STRIKING PLATE	1	PCS	5707292125859
83	82913	LEVER FOR WOMBAT	1	PCS	5707292125705
84	3501800	BLACK PVC CAP 5,5*30MM	1	PCS	5707292146762
85	2200020	RELEASE ARM	1	PCS	5707292143037
86	2200030	BUTTON FOR RELEASE ARM	1	PCS	5707292143051
88	80921	LOCKING PIN, WOMBAT	1	PCS	5707292111371
89	1310712	NYLON WASHER, NATURE	2	PCS	5707292356345
90	0663031	BUTTON HEAD 8*30, WOMBAT	8	PCS	5707292139610
91	1310710	NYLON WASHER	16	PCS	5707292140807
92	82933	BEARING BUSHING, WOMBAT	8	PCS	5707292125880
93	82946	END PIECE 15*30, WOMBAT	8	PCS	5707292126016
94	1750800	LOCK NUT, STAINLESS M8	8	PCS	5707292141484
95	82949-XX	BUMPER FOR WOMBAT	1	PCS	
96	1811200	SHEET METAL SCREW 3,5*13 PHX	1	PCS	5707292141712

XX	CHOOSE COLOUR CODE: 20: RED, 50: BLUE
----	---------------------------------------

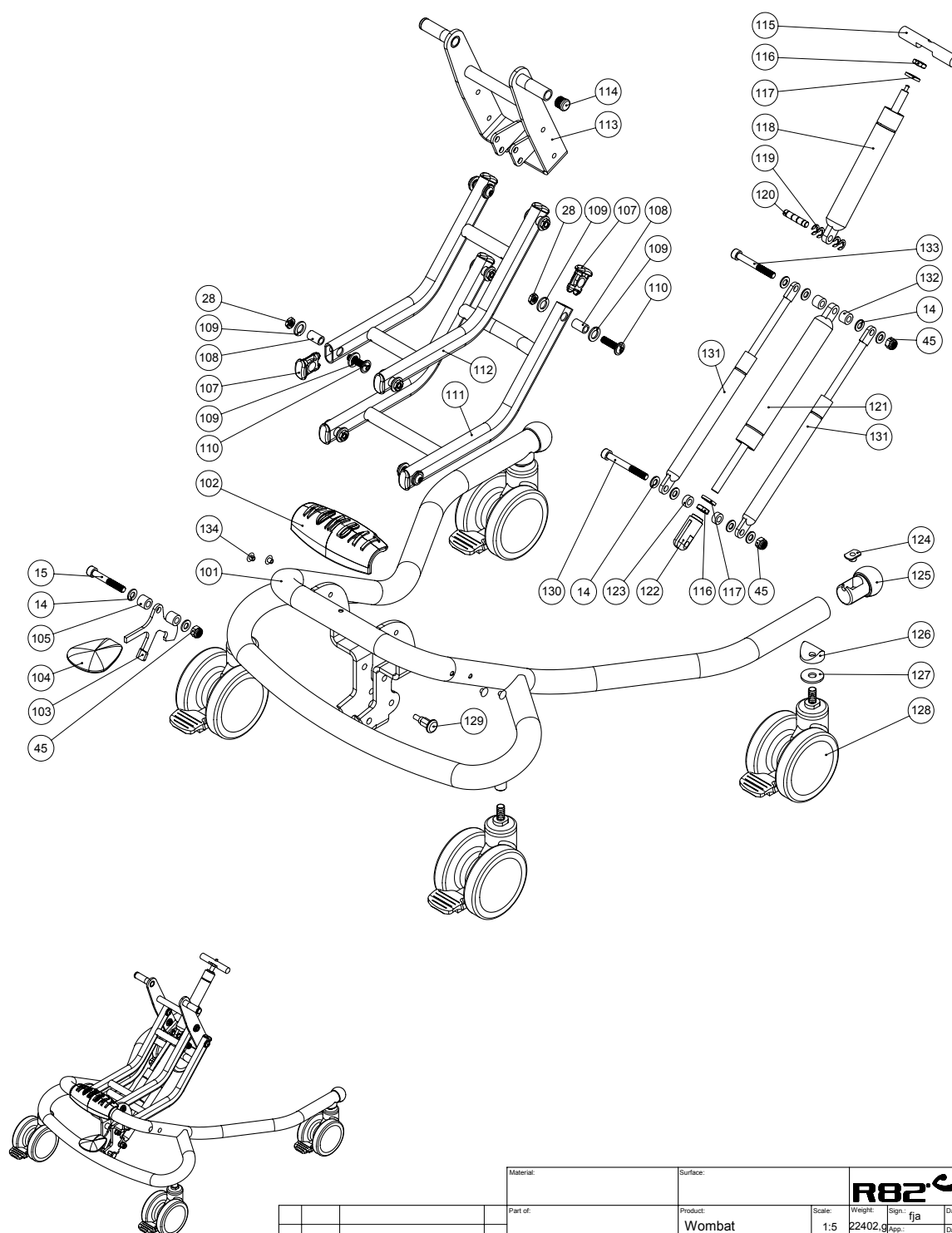


Material:	Surface:	R82[®] Parallelsvej 3 8751 Gedved Tlf.: 79 68 58 88 Fax: 79 68 51 13 Mail: r82@r82.dk			
Part of:	Product:	Scale:	Weight:	Sign:	Date:
	Wombat	1:5	22402,9	fja	29-06-2004
Title:	Format:	Drawing no.:			
Wombat 3 Frame	Custom	79197			

Position	No.	Description	Quantity	Unit of Measure	Ean code
14	1103200	FAZET WASHER 8MM	10	PCS	5707292140296
15	0603600	INSEX SCREW 8*55	1	PCS	5707292138262
28	1750800	LOCK NUT, STAINLESS M8	8	PCS	5707292141484
45	1607000	LOCK NUT 8MM, LARGE PACK	3	PCS	5707292141378
101	82725	FRAME, WOMBAT SIZE 3	1	PCS	5707292399311
102	82949-50	BUMPER FOR WOMBAT, BLUE	1	PCS	5707292232137
103	82914	PEDAL, WOMBAT	1	PCS	5707292125712
104	3470335	RELEASE BUTTON Ø8, 5*8MM	1	PCS	5707292146632
105	1350023	DISTANCE PIECE, BLACK	2	PCS	5707292140913
107	82946	END PIECE 15*30, WOMBAT	8	PCS	5707292126016
108	82933	BEARING BUSHING, WOMBAT	8	PCS	5707292125880
109	1310710	NYLON WASHER	16	PCS	5707292140807
110	0663031	BUTTON HEAD 8*30, WOMBAT	8	PCS	5707292139610
111	82790	SUPPORT BAR, WOMBAT SIZE 3	1	PCS	5707292400833
112	82921	SUPPORT BAR, WOMBAT	1	PCS	5707292125781
113	82782	TIPBRACKET	1	PCS	5707292398963
114	2501600	FERRULE D15*1,0 BLACK	2	PCS	5707292143969
115	82729	SHAFT	1	PCS	5707292400086
116	1761000	COUNTER NUT 10*1,0 FINE THREAD	2	PCS	5707292141538
117	1310712	NYLON WASHER, NATURE	2	PCS	5707292356345
118	2205255	BLOC-O-LIFT 5085UX 500N FAST	1	PCS	5707292398154
119	1145070	LOCK WASHER 7MM	4	PCS	5707292140586
120	80924	Ø8*40 SPEC LOCKING PIN,WOMBAT	1	PCS	5707292111401
121	2205300	BLOC-O-LIFT GASSPRING 5251HS	1	PCS	5707292143389
122	6118822	HORIZONTAL RELEASE	1	PCS	5707292186744
123	1350000	DISTANCEPART BLACK 6*15*8,2 MM	2	PCS	5707292140876
124	1600525	STAINL. WELDING NUT W/FLANGE	2	PCS	5707292141149
125	82829-50	BALL Ø40, BLUE	2	PCS	5707292231871
126	82894	DISTANCE PIECE 30*3,4*10,4	2	PCS	5707292140951
127	1108050	WASHER M10, DIN9021	2	PCS	5707292140401
128	3006930	100 MM GREY DOUBLE WHEEL	4	PCS	5707292145918
129	0380183	SPEC. HOUSE FOR LOCK, COMPLETE	1	PCS	5707292136046
130	0603700	INNSEX SCREW 8*70	1	PCS	5707292138293
131	2202100	LIFT-O-MAT 5078UO 250N	2	PCS	5707292400468
132	1350021	DISTANCE PIECE 11*15*8,2MM	2	PCS	5707292140890
133	0603665	INSEX SCREW 8*65	1	PCS	5707292138286
134	1310540	COVERING PUSHBUTTON, BLACK	4	PCS	5707292140692

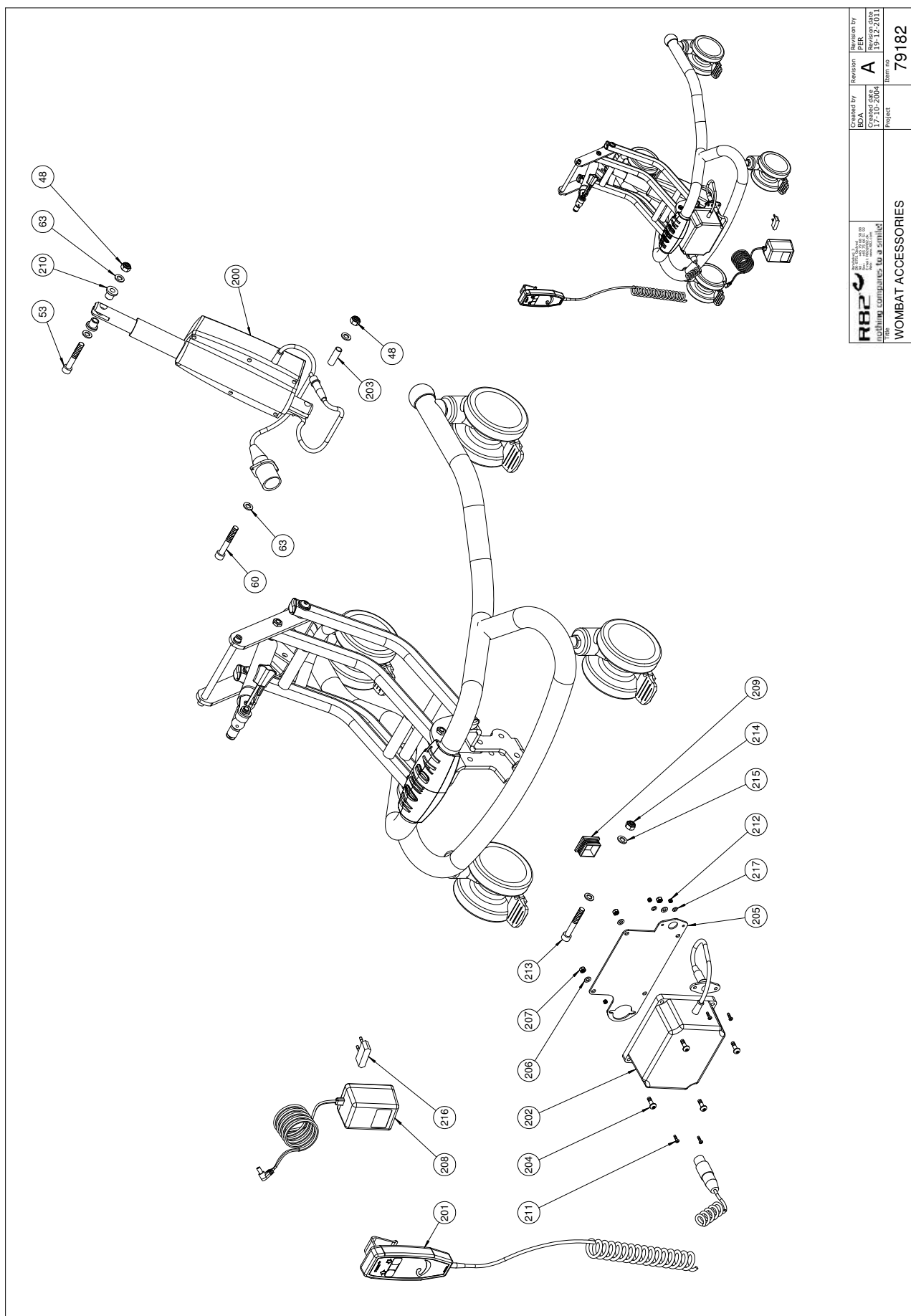

 nothing compares to a smile! Drawn by R82 Created date 17-10-2004 Revision A Created date 19-12-2011	Created by BDA	Revision A	Revision by PER
	Created date 17-10-2004	Created date 19-12-2011	
Title WOMBAT ACCESSORIES	Project 79182	Item no 79182	

Position	No.	Description	Quantity	Unit of Measure	Ean code
48	1607000	LOCK NUT 8MM, LARGE PACK	3	PCS	5707292141378
53	0603550	INSEX SCREW 8*50	1	PCS	5707292138255
60	0603600	INSEX SCREW 8*55	2	PCS	5707292138262
63	1103200	FAZET WASHER 8MM	7	PCS	5707292140296
200	2222133	ACTUATOR LA28 F/WOMBAT	1	PCS	5707292492272
201	2220600	HANDSET W/R82 PRINT & CHARGING	1	PCS	5707292492241
202	2222110	BATTERIBOKS BA18027	1	PCS	5707292492258
203	6108200	PVC PIPE PA11 Ø10/Ø8 MM	0,03	METER	5707292185655
204	1902760	CROSS-HEADED SCREW 5*16 MM	4	PCS	5707292142146
205	77576	BRACKET FOR BATTERY	1	PCS	5707292125477
206	1102900	FAZET WASHER 5 MM	4	PCS	5707292140272
207	1606300	LOCK NUT 5 MM	4	PCS	5707292141354
208	2222160	CHARGER WO/PLUG	1	PCS	5707292492296
209	2505000	FERRULE E30*301,0-2,0 INT.BLAC	1	PCS	5707292144140
210	82848	BUSH	2	PCS	5707292125484
211	1902590	CROSSHEAD SCREW M3*10, FZB	4	PCS	5707292408389
212	1606250	SELF-LOCKING NUT 3MM	4	PCS	5707292408396
213	0603550	INSEX SCREW 8*50	1	PCS	5707292138255
214	1607000	LOCK NUT 8MM, LARGE PACK	1	PCS	5707292141378
216	2222165	PLUG F/DK	1	PCS	5707292492388
216	2222166	PLUG F/USA+CANADA+TAIWAN	1	PCS	5707292492395
216	2222167	PLUG FOR UK	1	PCS	5707292492401
216	2222169	PLUG F/NEW ZEALAND + AUS	1	PCS	5707292492425
217	1102800	FAZET WASHER 4 MM	1	PCS	5707292140265

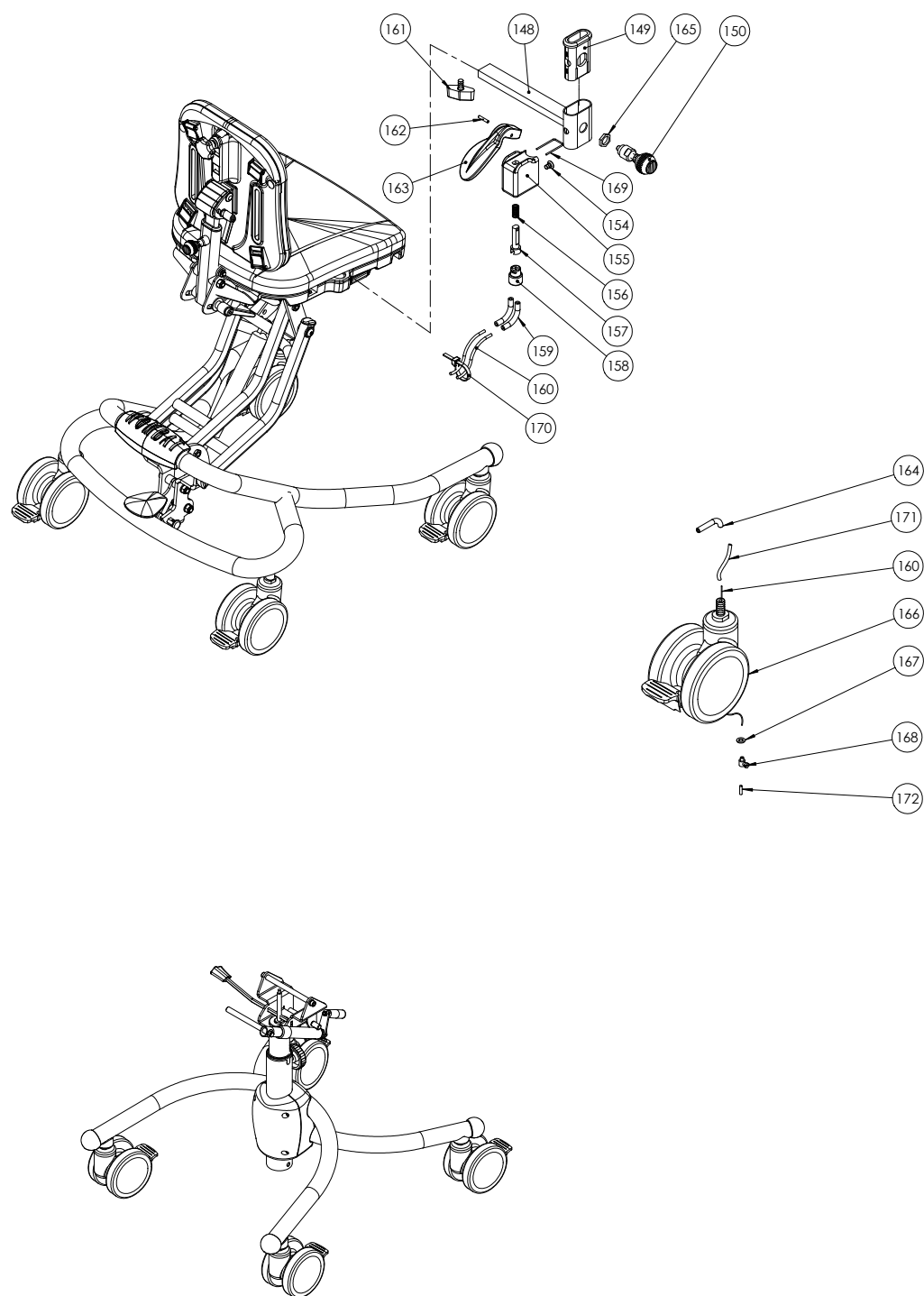


Printed: 19-01-2012

				Material:		Surface:			
--	--	--	--	-----------	--	----------	--	---	--

Position	No.	Description	Quantity	Unit of Measure	Ean code
148	82870	MONTAGEBESLAG F/ARMLÆN,WOMBAT	1	PCS	5707292355478
149	82868	PLASTBØSNING 10/30, WOMBAT CB	1	PCS	5707292355454
150	9500001	LOCK WOMBAT	1	PCS	5707292404527
154	0631809	INSEX UH. M5*8	1	PCS	5707292353337
155	70127	HOLDER F.BRAKE,HANDBRACE,TIGER	1	PCS	5707292101600
156	0310129	COMPRESSION SPRING FZB 1,5X12U	1	PCS	5707292438782
157	6108148	PIN, BRAKE	1	PCS	5707292185501
158	70139	PROP	1	PCS	5707292101648
159	0389000	VINKLET KABELSTRAMMER	2	PCS	5707292136152
160	0371850	CABLE F.CENTRALBREMSE WOMBAT BASIC	2	PCS	5707292135872
161	3652500	WING SCREW 28*11 M6*16MM BLACK	1	PCS	5707292147172
162	0588014	RUSTL.TIGHTENING SPRING 3*14	1	PCS	5707292137753
163	70125	HANDLE FOR BRAKE, TIGER	1	PCS	5707292101587
164	82909	KABELFØRING	2	PCS	5707292125668
165	1785000	NUT 12MM HEXAGON, FZB	1	PCS	5707292141583
166	3006950	100 MM DOUBLE WHEEL W/CENTRAL	2	PCS	5707292145925
167	1102800	FAZET WASHER 4MM	2	PCS	5707292140265
168	0388010	SKRUENIPPEL 7*7MM	2	PCS	5707292136121
169	6108146	SAFETY BRACE Ø2	1	PCS	5707292185495
170	0235075	CABLE TAPE, BLACK	1	PCS	5707292135308
171*	5002750	CABLE FOR BRAKE, SIZE 1+2	1	PCS	5707292413635
	5002965	CABLE FOR CENTRAL BRAKE SIZE 3	2	PCS	5707292408846
172	0387000	END NIPPLE FOR WIRE	2	PCS	5707292136091


*		CHOOSE SIZE
---	--	-------------

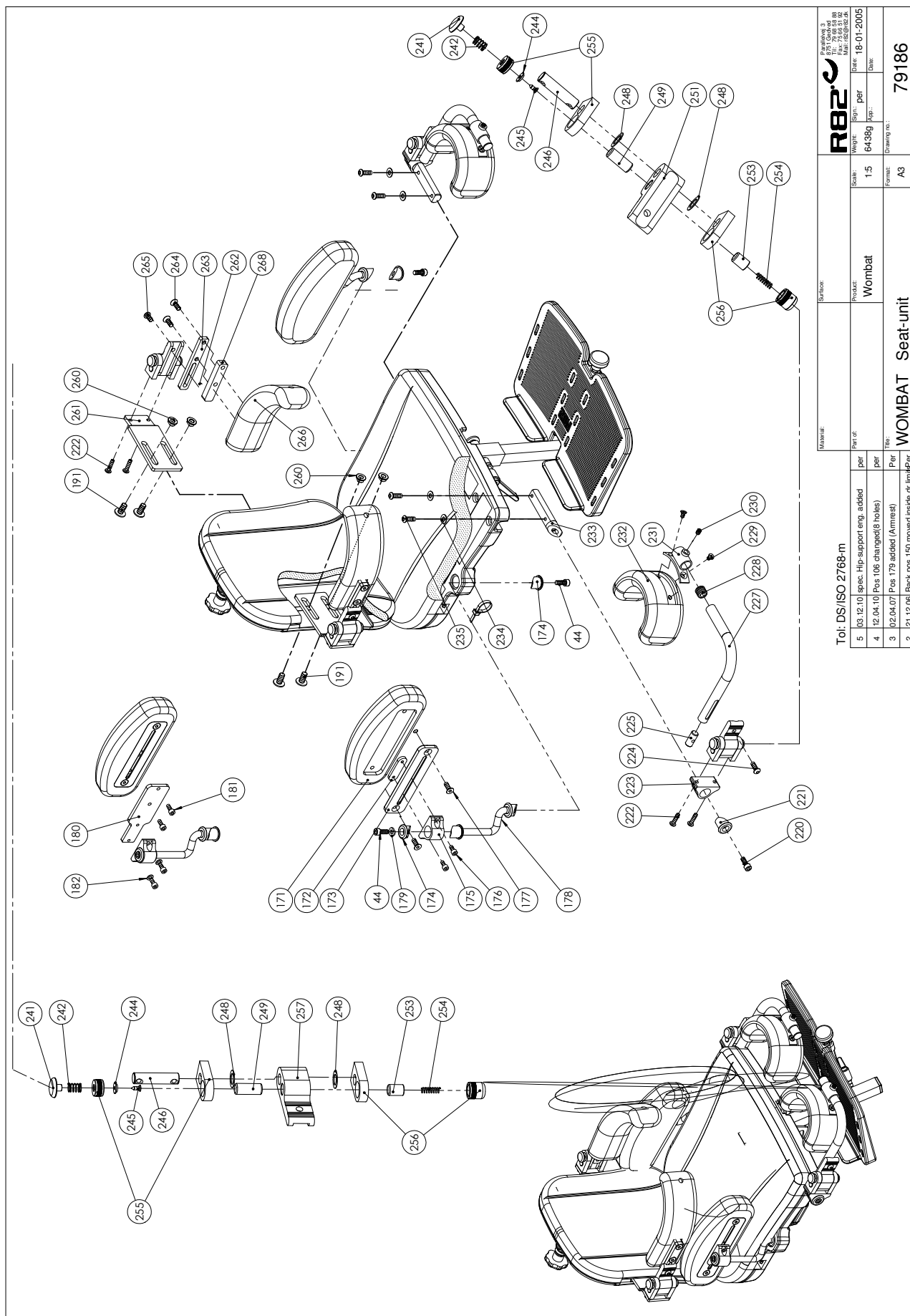

Material:		Surface:		R82		Parallelvej 3 1710 75 66 51 92 Tlf: +45 75 66 51 92 Fax: +45 75 66 51 92 E-mail: R82@R82.dk Web: www.R82.com		Date: 18-01-2005		Drawing no.: 79186	
Part of:		Product:		Weight:		Scale:		Date:		Drawing no.:	
per		Wombat		6438g		1:5		Date:		Drawing no.:	
per		Tol: DS/ISO 2768-m		5 03.12.10 spec. Hip support eng. added		4 12.04.10 Pos 106 changed (8 holes)		3 02.04.07 Pos 179 added (Arrest)		2 21.12.06 Back pos 150 moved inside of arm	
per		WOMBAT Seat-unit		per		per		per		per	

Position	No.	Description	Quantity	Unit of Measure	Ean code
44	0602202	INSEX M6*18, FZB	4	PCS	5707292459343
171*	6175171	CUSHION SIZE 1	2	PCS	5707292189301
	6175172	CUSHION SIZE 2	2	PCS	5707292189318
172	82974	THREADED PLATE FOR HIP SUPPORT	2	PCS	5707292126184
173	82953	PLATE FOR HIP SUPPORT	2	PCS	5707292126047
174	82939	TIGHTENING BUSHING F. SWINGARM	4	PCS	5707292125941
175	82542	HOUSE TOOL	2	PCS	5707292124593
176	0658010	BUTTON HEAD M5 X 10,STAINLESS	4	PCS	5707292139375
177	0609728	INSEX, STAINLESS 6*16 UH	4	PCS	5707292138491
178	74870	STEERING ARM, SIDESUPPORT	2	PCS	5707292419101
179	1100900	SPRING WASHER M6 FZB, 6.1/9.9	4	PCS	5707292459336
220	0602200	INSEX SCREW 6*16	2	PCS	5707292138095
221	82939	TIGHTENING BUSHING F. SWINGARM	2	PCS	5707292125941
222	0658020	BUTTON HEAD M5 x 20,STAINLESS	4	PCS	5707292139405
223	82542	HOUSE TOOL	2	PCS	5707292124593
224	0659500	BUTTON HEAD SCREW	2	PCS	5707292139450
225	82935	GEVINDPAL	2	PCS	5707292125903
227*	82374	CURVED KNEE BRACE WOMBAT 1+2	2	PCS	5707292123756
	82696	CURVED KNEE BRACE WOMBAT 3	2	PCS	5707292402363
228	2501600	FERRULE D15*1,0 BLACK	2	PCS	5707292143969
229	0631860	INSEX 5*10, FZB	4	PCS	5707292139191
230	0700906	STOP SCREW M8*6, RAW	2	PCS	5707292387585
231	82371	HOLDER FOR PELOT	2	PCS	5707292123725
232	6175161	KNEESUPPORT SIZE 1 PANDA	2	PCS	5707292189288
233	82938	TIGHTNING SPINDLE FOR KNEESUPP	2	PCS	5707292125934
234	1103000	FAZET WASHER 6MM	4	PCS	5707292140289
235	0659500	BUTTON HEAD SCREW	4	PCS	5707292139450
241	82559	RELEASE BUTTON	2	PCS	5707292124760
242	0308710	RF. SPRING 0.5*8*10MM*4V	2	PCS	5707292135483
244	6108350	WASHER 10*2,4	2	PCS	5707292185884
245	0683250	STAINLESS INSEX M3*8	2	PCS	5707292139672
246	82556	AXLE	2	PCS	5707292124739
248	1310640	WASHER, NATURE Ø16*10*0,5MM	4	PCS	5707292140760
249	82554	UPPER PIN	2	PCS	5707292124715
251	82541	ADAPTOR HOUSE FOR KNEE SUPPORT	1	PCS	5707292124586
253	82555	LOWER PIN	2	PCS	5707292124722
254	0309175	COMP. SPRING 0,8*5,8*17,5MM*10	2	PCS	5707292135490
255	5001080	UPPER RELEASE HEAD WITH HOLDER	2	PCS	5707292355690
256	5001081	LOWER RELEASE HOUSE W/HOLDER	2	PCS	5707292355706
257	82541	ADAPTOR HOUSE FOR KNEE SUPPORT	1	PCS	5707292124586

*		CHOOSE SIZE
---	--	-------------

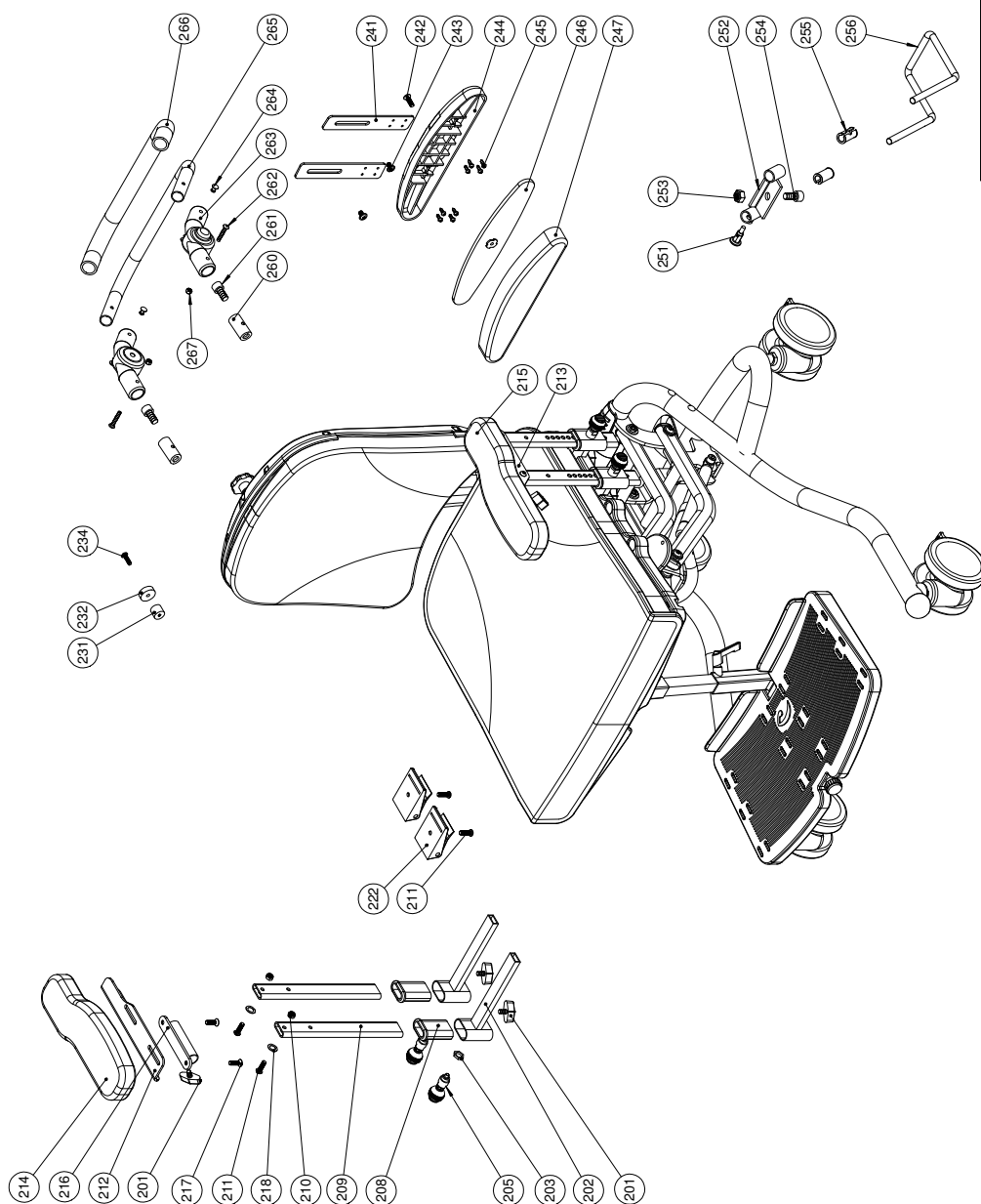

Material:		Surface:		<	
-----------	--	----------	--	--	--

Position	No.	Description	Quantity	Unit of Measure	Ean code
201	3652501	R82 HANDLE M6*16MM SCREW	6	PCS	5707292413963
202	82960	MOUNTFITTINGS F/ARMREST,WOMBAT	4	PCS	5707292126078
203	1785000	NUT 12MM HEXAGON, FZB	4	PCS	5707292141583
205	9500001	LOCK WOMBAT	4	PCS	5707292404527
208	82948	BUSH 10/30, WOMBAT	4	PCS	5707292126023
209	82963	ARM REST, WOMBAT 3	4	PCS	5707292126092
210	1606500	LOCK NUT 6 MM	4	PCS	5707292141361
211	0641620	BUTTON HEAD SCREW 6*20	4	PCS	5707292139276
212	82718	ARMREST BRACKET LEFT SIZE 3	1	PCS	5707292442727
213	82705	ARMREST RIGHT SIZE 3	1	PCS	5707292442734
214*	6175154	PUR ARMREST SIZE 2&3, RIGHT	1	PCS	5707292189240
	6175163	ARMREST PU. WOMBAT SZ 4. RIGHT	1	PCS	5707292443151
215*	6175156	PUR ARMREST SIZE 2&3, LEFT	1	PCS	5707292189257
	6175164	ARMREST PU. WOMBAT SZ 4. LEFT	1	PCS	5707292443168
216	82975	MOUNTING FOR TRAY, WOMBAT	2	PCS	5707292126191
217	0611000	INSEX SCREW COUNTERS. 6*20	4	PCS	5707292138682
218	1310640	WASHER, NATURE Ø16*10*0,5MM	4	PCS	5707292140760
222	0442070	FIXLOCK CLAMP 40 MM W/HOLE	4	PCS	5707292136398
231	82762	BUSH	2	PCS	5707292401786
232	82761	DISK	2	PCS	5707292401793
233	0442065	FIXLOCK BUCKLE 25MM BLACK	2	PCS	5707292136381
234	0658020	BUTTON HEAD M5 x 20,STAINLESS	2	PCS	5707292139405
241	82744	MOUNTINGPLATE	2	PCS	5707292398901
242	0612500	INSEX SCREW COUNTERS. 6*25	1	PCS	5707292138712
243	0659010	BUTTON HEAD WITH FLANGE M6*10	2	PCS	5707292356819
244	82748	LUMBAR SUPPORT	1	PCS	5707292400093
245	1940033	ECO-SYN Z M3,5*10 RUSTFRI	8	PCS	5707292422965
246	5700483	UPHOLSTERED LOWER BACK SUPP	1	PCS	5707292399304
247	5700484	COVER FOR LUMBAR SUPPORT,BLACK	1	PCS	5707292399427
251	0380182	LOCKING HOUSE, COMPLETE M10*1	1	PCS	5707292357953
252	80343	Guide HIGH-LOW	1	PCS	5707292469588
253	1607500	LOCK NUT 10 MM	1	PCS	5707292141385
254	0604000	INSEX SCREW 10*20	1	PCS	5707292138361
255	82375	PVC-BUSHING FOR PELOT	2	PCS	5707292123763
256	80342	Brace for anti-tip safety	1	PCS	5707292469595
260	80705	BUSH FOR HANDBRACE	2	PCS	5707292436900
261	0604000	INSEX SCREW 10*20	2	PCS	5707292138361
262	0640930	BUTTON HEAD M5*30 FZB	2	PCS	5707292139221
263	2523000	ADJUSTMENT FOR HANDBRACE 20MM	2	PCS	5707292354617
264	0506513	STAINLESS BLIND RIVET Ø4, 8*12	4	PCS	5707292137326
265	80706	TUBE FOR HANDBRACE	1	PCS	5707292436917
266	2500550	GRIP 700MM LENGTH FOR Ø20MM	1	PCS	5707292388520
267	1606300	LOCK NUT 5 MM	2	PCS	5707292141354

*	CHOOSE SIZE
---	-------------